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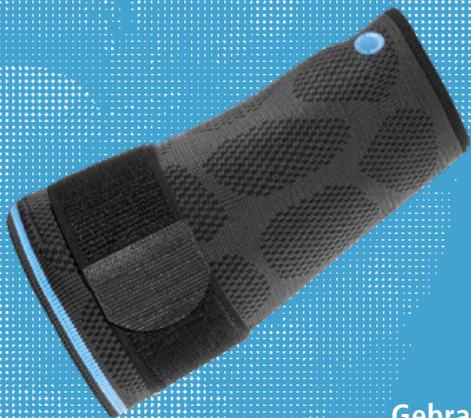
Instructions for use

Epicondylitisbandage

Epicondylitis Support

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Lumbalbandage
Lumbar Support

Kniebandage
Knee Support

Achillesbandage
Achilles Tendon Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex[®] comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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Knöchelbandage
Ankle Support

Epicondylitisbandage
Epicondylitis Support

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Introduction

Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

To put on the support, open the elastic strap. Pull the wider end of the support over the forearm, not over the elbow. With tennis elbow the pad must lie on the dorsal side of the forearm, and with golfer's elbow it must lie on the ventral side. Please wear the support directly on the skin – this is the only way to obtain the desired effect.

2

The separate Velcro strap can be worn in order to obtain a better fit. Attach the narrow hook strip of the strap to the loop strip on the support. Now lead the strap around the arm and fasten it loosely. Please ensure that the product is secure but not too tight in order to avoid pinching.

Purpose

The Dynamics Plus epicondylitis support is worn on the forearm and relieves strain on the elbow tendon insertions. The compressive effect works together with the large pad help to absorb vibrations caused by the wearer's muscles or any external forces.

Important notes

- The initial fitting and positioning of the product as well as training in its proper, safe use should be carried out by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product is fitted correctly. Check the product to ensure it is working and check it for wear and damage (such as breaks, cracks or distortion) before each use. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Before washing, remove the pad from the support and close the Velcro fastener of the strap. The pad must not be machine washed. Please clean the pad with a damp cloth or handwash with a mild soapy solution.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent)

and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.

- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to

the skin in the usage area. These substances can affect the material of the product.

- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

Delicate wash cycle

Do not bleach

Do not tumble dry

Do not iron

Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- Tennis elbow (epicondylitis humeri radialis)
- Golfer's elbow (epicondylitis humeri ulnaris)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using this product:

- Skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- Feeling of numbness and circulatory problems in the area of application
- Problems with lymphatic drainage as well as unexplained swelling

Material composition

45 % Polyamide

41 % Silicon

7 % Elastane

4 % PU

3 % Polyester

Aluminium

Disposal

Please dispose of the product according to local regulations after the end of use.

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Guarantee

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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Ofa Bamberg GmbH