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Instructions for use

Knieorthese Classic

Knee Orthosis Classic

dynamics[®]

ofa bamberg

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We have a lot to offer

the medical
compression
specialist
since
1928
made
in Germany

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Ofa Bamberg.

**More than 90 years' experience
with your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy. In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive support and travel stockings.

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Fitting of the knee orthosis

(only by experts)

To adjust the joints, loosen the screws on the joint cover using the screwdriver provided (→ see Figure A).

The wedges for limiting flexion (0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°) are inserted into the joints at the narrow, round groove (→ see Figure B), the wedges for limiting extension (0°, 10°, 20°, 30°, 40°) at the wide, square groove (→ see Figure C). The screws are then tightened again.

Both joints should always be set to the same levels. Flexion and extension wedges with the same angular degrees are used to lock the joints.

To make minor adjustments to the fit, the hinges above the joints can be readjusted using the Allen key provided (→ see Figure D).

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Introduction

Dear user, we are delighted that you have chosen an Ofa Bamberg product and we want to thank you for putting your trust in us. Please take a moment to read this product information through carefully to ensure your Dynamics product works as effectively as possible.

Ofa Bamberg wishes you a speedy recovery!

← Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

1

To put the knee orthosis on, open the Velcro straps on the rounded end and pull them out of the eyelet. When putting the orthosis on, please be aware that it gets narrower at the lower end. Put the orthosis against the front of your slightly bent leg. It is correctly positioned if the centre of the joint is in line with the lower edge of the kneecap.

2

Ensure that you close the Velcro straps in the right order. Close the upper strap on the back of the lower leg first (1) by inserting it through the eyelet provided and fastening it tightly. Then close the lower strap on the back of the thigh (2) and then the lower one on the back of the lower leg (3).

3

Then close the front Velcro straps on the shin (4) and the thigh (5) and finally the upper strap on the back of the thigh (6). Ensure that the product is secure but not too tight so as to prevent pinching.

Intended Use

The Dynamics Knee Orthosis Classic is designed to stabilise and immobilise your knee joint in part or in full. The solid aluminium frame and an adjustable joint hinge limit the range of movement of your knee joint, helping to support its healing process.

Important information

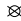
- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists. → See page 5.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

english

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Detach the removable padding and Velcro straps from the splints for washing. Ensure that you do not pull the Velcro strips off the splint too. The splint can be cleaned with a damp cloth.
- Close all of the Velcro fasteners and wash the fabric by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- For optimum care of the joints, they can be sprayed occasionally with a dry lubricant (Teflon spray).
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Hand wash
 -  Do not bleach
 -  Do not tumble dry
 -  Do not iron
 -  Do not dry clean
- Do not use fabric softener!**

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

- Instability after injuries to the anterior cruciate ligament and/or the lateral ligaments
- Long-term treatment of torn ligaments with lasting instability
- Postoperative stabilisation after ligament surgery

Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

Material composition

40 % Aluminium alloy
 20 % Polyester
 20 % Polyamide
 10 % Polyurethane
 3 % Zinc
 2 % Acetal
 2 % Polypropylene
 1 % Styrene-Butadiene Rubber
 1 % Iron
 1 % PVC

Disposal

Please dispose of the product according to local regulations after the end of use.

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Garantie

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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dynamics®
**We'll get you moving
again in no time.**

aus 80 %
recycltem Papier

ofa bamberg

Ofa Bamberg GmbH

Laubanger 20

D-96052 Bamberg 

Tel. + 49 951 6047-0

Fax + 49 951 6047-185

info@ofa.de

www.ofa.de

ofa austria

Ofa Austria

Wasserfeldstr. 20

A-5020 Salzburg

Tel. + 43 662 848707

Fax + 43 662 849514

info@ofaaustria.at

www.ofaaustria.at