
Daumenorthese Thumb Orthosis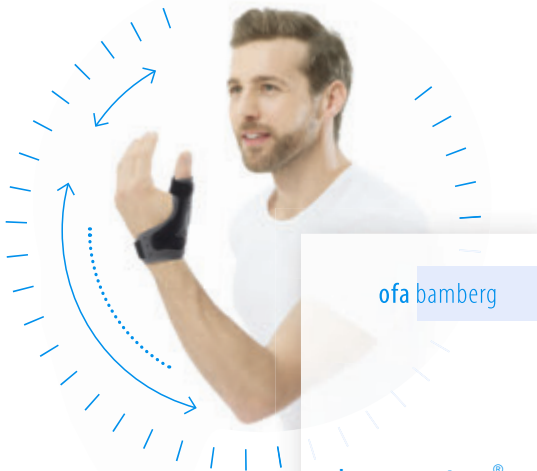

ofa bamberg

Ofa Bamberg GmbH
Laubanger 20
D-96052 Bamberg
Tel. +49 951 6047-0
Fax +49 951 6047-185
info@ofa.de
www.ofa.de

ofa austria

Ofa Austria
Wasserfeldstr. 15
A-5020 Salzburg
Tel. +43 662 848707
Fax +43 662 849514
info@ofaaustria.at
www.ofaaustria.at

ofa bamberg

dynamics®

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Ofa Bamberg.
More than 90 years'
experience for your
health.

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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Important notes
Thumb orthosis

← **Fitting instructions**

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

To put on the orthosis, open the Velcro fastener at the wrist. Slide your hand into the orthosis and close the Velcro fastener by pulling it through the provided eyelet and securing it.

2

Now adjust the Velcro fastener at the thumb in such a way that the thumb is securely held in the orthosis. Please ensure that the product is secure but not too tight in order to avoid pinching.

Dear user,

The Dynamics thumb orthosis relieves the metacarpal joint and basal joint of the thumb by means of targeted movement restriction. The palm of the hand and the other fingers remain freely movable. The incorporated aluminium reinforcement can be optimally adapted to your hand.

Ofa Bamberg wishes you a speedy recovery!

Please note

- **The initial adjustment and strapping on of the product must be carried out by qualified specialist staff.**
- If the product has to be worn in combination with other products, then please discuss this in advance with your doctor.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance. However, should there still be reasons for complaint, please contact your specialist supplier.

Important notes

Thumb orthosis

Care instructions

- The product is suitable for immediate use. Our recommendation: wash before wearing for the first time.
- Hand-wash the product at a maximum temperature of 30 °C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Indications

- sprains, bruising to thumb
- injuries to the capsule/ligament apparatus
- immobilisation of thumb after operation (e.g. with skier's thumb)
- signs of wear of the articular surfaces (e.g. rhizarthrosis)
- chronic instability of the metacarpal joint or basal joint of the thumb

Material composition

- 46 % polyamide
- 46 % polyurethane
- 7 % cotton
- 1 % elastane
- Splint: duraluminium

Care

-  Hand wash
 -  Do not bleach
 -  Do not put into a dryer
 -  Do not iron
 -  Do not dry clean
- Do not use softener!

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using this product:

- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- feeling of numbness and circulatory problems in the area of application
- problems with lymphatic drainage as well as unexplained swelling

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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