## Carpalorthese Carpal Orthosis

ofa bamberg

**Ofa Bamberg GmbH**  
Laubanger 20  
D-96052 Bamberg  
Tel. +49 951 6047-0  
Fax +49 951 6047-185  
info@ofa.de  
www.ofa.de

ofa austria

**Ofa Austria**  
Wasserfeldstr. 15  
A-5020 Salzburg  
Tel. +43 662 848707  
Fax +43 662 849514  
info@ofaaustria.at  
www.ofaaustria.at

dynamics®

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---

---

EN english	<b>Carpal orthosis</b>	Adjustment by experts	6
		Important notes	10
		Warranty	20

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### **Ofa Bamberg.** **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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---

### **Moulding the synthetic splint** (by healthcare professionals only)

The synthetic splint is thermoformable and can be moulded to your requirements by trained staff. This requires the inner padding and the Velcro tapes to be removed. The parts to be moulded are heated evenly with a hot air blower at a temperature of about 200° C. The splint can now be moulded into the required shape and then cooled. The material can be reheated and reshaped, if necessary.

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---

### Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

### Fitting instructions

---

**The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.**

1

---

To put on the orthosis, open all Velcro fasteners and place your forearm in the splint. Move your fingers and thumb carefully until your hand has reached an optimum position.

2

---

First close the Velcro tape around your wrist, then the Velcro tapes around your finger and thumb, and finally the Velcro tape for the forearm. Please ensure that the product is secure but not too tight, in order to prevent pinching.

3

---

The additional strap available for the back of the hand can also be fastened. The padding should cover the entire surface of the back of the hand. Pass the Velcro tape around your hand and fasten it by pulling it through the eyelet and then closing it.

---

**The Dynamics carpal orthosis** provides optimum immobilisation of the wrist and hand. The finger support keeps the hand in a functional position so that physiotherapy exercises can be carried out while wearing the orthosis. The open design allows for the orthosis to be easily put on and taken off, thus simplifying personal hygiene and wound treatment easier.

---

### Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff.  
→ see page 6
- If the product has to be worn in combination with other products, then please discuss this in advance with your doctor.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then contact your doctor or specialist supplier immediately.
- We use a comprehensive quality management system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

## Care instructions

- The product should be washed before being used for the first time.
- Detach the removable padding and Velcro tapes from the splint before washing. Please be sure to avoid pulling off the Velcro strips on the splint. The splint can be cleaned with a moist cloth.
- Wash the textiles by hand at a maximum temperature of 30° C. Use mild soap suds and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

## Indications

- severe sprains
- wrist degeneration (arthrosis)
- severe tendon sheath inflammation (tendovaginitis)
- nerve damage in hand area
- after-treatment of fractures, joint injuries (dislocations) and operations
- paralysis following a stroke (hemiplegia)

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

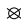
## Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - feeling of numbness and circulatory problems in the area of application
  - problems with lymphatic drainage as well as unexplained swelling

## Material composition

Synthetic splint:  
100 % polyethylene  
Inner padding:  
100 % polyethylene  
Velcro tapes:  
100 % polyester

## Care

-  Hand wash
  -  Do not bleach
  -  Do not put into a dryer
  -  Do not iron
  -  Do not dry clean
- [Do not use softener!](#)

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---

### Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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