gilofa®

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84 % Polyamid
16 % Elasthan

Dear User,

Gilofa travel and support stockings are the result of decades of experience in the development of medical compression stockings. You have decided to choose a high quality health product. In addition to medical aspects, with Gilofa we pay particular attention to the wearing comfort. Gilofa stockings feel pleasant, are comfortable to wear and attractive in appearance. Healthy living made easy. Whether travelling, participating in sports or simply because you wish to do yourself some good in everyday life.

Important information

The stockings must be obtained from a medically trained specialist retailer. Your retailer will also advise you on how to correctly put on your stockings. If you experience leg pain or skin irritation while wearing these stockings, please consult your doctor immediately. We test our products within the scope of a comprehensive quality assurance. However, if you still have cause for complaint, please contact your specialist retailer. Do not attempt to perform repairs yourself. Return the stockings to the specialist retailer who sold them to you. Any pilling of the material that may arise due to friction with other garments will not reduce the stocking's quality.

Warranty

The lifetime of compressive stockings is limited to 5 years when stored properly. With regular use and proper care, we guarantee optimal compressive effect for 6 months. After this period, the products are excluded from the warranty. Defects attributable to normal wear and tear or improper use are not covered by the guarantee. Improper use also includes any subsequent modification to the product that has not been carried out by the manufacturer. The product is intended for single use only. If it is passed on to other persons for reuse, the manufacturer's product liability is no longer valid.

Effective principle of compression stockings

The pressure decreases from the bottom to the top and promotes blood circulation and supports the venous system.

Indications / Intended use

Feeling of heaviness and tiredness in the legs. Prophylactic effect against vein complaints and deep vein thrombosis caused by prolonged travel in people with healthy veins. Consult your doctor if in doubt as to whether there is any indication of venous disease that requires treatment.

Contraindications

- advanced peripheral arterial occlusive disease.
- congestive heart failure
- septic phlebitis
- Phlegmasia cerulea dolens

Risks to consider:

- marked weeping dermatoses
- allergies to compression stocking materials
- severe sensitivity disorders of the extremity
- advanced peripheral neuropathy (e.g. diabetes mellitus)
- primary chronic polyarthritis

If you have any questions, please ask your doctor.

Risks and side effects

Users with sensitive skin may experience itching and skin irritation when using compression stockings. Adequate skin care is therefore recommended. If you suffer the following symptoms, please remove the stockings immediately and consult your doctor before continuing using them: blue or white coloration of the toes, sensations of discomfort and numbness, increasing pain, shortness of breath and sweating, acute movement restrictions.

Disposal

After the end of the usage period, please dispose of the product in accordance with local regulations.

Mandatory reporting

Due to legal regulations within the EU, patients and users are obligated to report any serious events with the use of a medical device to both the manufacturer and the competent national authority (in Germany BfArM) without any delay.

Care instructions

The product is suitable for immediate use. We recommend washing the product before wearing it for the first time. Then wash daily by hand or on a delicate fabric program. Wash the stockings separately with mild detergent without fabric softener (e.g. "Ofa Clean special detergent") at a maximum temperature of 40°C and rinse them well. The stockings can be spun gently, but do not wring them out. Dry your stockings in air or in a dryer on a gentle cycle. Do not place them on a heater or in direct sunlight. Never iron the stockings.

To maintain the effectiveness of your stockings, please do not use chemical cleaning agents, petrol, or fabric softeners. These substances damage the knitted fabric. Compressive stockings are largely resistant to fats and oils, but should not come into contact with ointments. Stockings with silicone-coated grip tops should generally not come into contact with solvents, ointments and re-greasing washing lotions and body creams. Otherwise, the adhesion of the grip tops could be permanently impaired. Store the stockings in a dry place away from sun and heat, in the original packaging if possible.

Wash inside out and do not use fabric softener!

-  Wash on a delicate or gentle cycle
-  Do not bleach
-  Dry gently and carefully
-  Do not iron
-  No chemical dry cleaning

Tips for putting the stockings on

Carefully pull your stockings/pantyhose up your leg from the bottom to the top, do not pull them up from the top edge. Please avoid any damage that could be made by jewellery or long fingernails.

Material composition

gilofa®

with cotton

50% cotton
40% polyamide
10% elastane

gilofa®

with aloe vera

Continuous skin care. Special microcapsules dispense in a controlled manner the aloe vera extracts to the skin for more than 100 washings.
42% polyamide (microfibre with aloe vera)
35% polyamide
15% cotton
8% elastane

gilofa®

with microfibres

53% polyamide
24% polyamide (microfibre)
13% cotton
10% elastane

gilofa® fine

52% polyamide (with almond and rose oil)
48% elastane

gilofa® style

52% polyamide (microfibre)
29% elastane
19% polyamide

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Ofa Bamberg GmbH