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Instructions for use

Kniebandage
Knee Support

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Arthroflex® Komfortzone
in der Kniekehle

ofa bamberg

dynamics®
plus

Anziehanleitung in Bildern

Fitting instructions
in pictures

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dynamics[®] plus
Lumbalbandage
Lumbar Support

dynamics[®] plus
Handgelenkbandage
Wrist Support

dynamics[®] plus
Epicondylitisbandage
Epicondylitis Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex[®] comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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Introduction

Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

When putting on the support, please note that it becomes narrower at the lower end. Pull the support over your leg at its wider end first, with the silicone pad at the front. The support is sitting correctly if the silicone ring encloses the kneecap.

Purpose

The Dynamics Plus Knee Support

relieves your knee joint by means of its compressive effect. The Plus pad (silicon pad) with extensions on the thigh and the shin provides an increased massage effect around the kneecap. For optimum wearing comfort the extremely thin, highly elastic Arthroflex® comfort zone is incorporated in the sensitive area of the back of the knee.

 **Important notes**

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Wear the product directly on your skin. It may only be worn on intact skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

- Delicate wash cycle
- Do not bleach
- Do not tumble dry
- Do not iron
- Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- Kneecap cartilage disorder (chondromalacia patellae)
- Contusions, sprains (distortions)
- Meniscus damage
- Degenerative changes, such as joint inflammation (arthritis) and joint degeneration (arthrosis)
- After knee joint operations

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- Skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - Feeling of numbness and circulatory problems in the area of application
 - Problems with lymphatic drainage as well as unexplained swelling

Material composition

41 % Polyamide
 25 % Silicone
 23 % Metal
 6 % Elastane
 3 % Viscose
 1 % Polyester
 1 % Cotton
 PU
 Aluminium

Disposal

Please dispose of the product according to local regulations after the end of use.

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Garantie

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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Ofa Bamberg GmbH