## Knierthese Knee Orthosis

ofa bamberg

**Ofa Bamberg GmbH**  
Laubanger 20  
D-96052 Bamberg  
Tel. + 49 951 6047-0  
Fax + 49 951 6047-185  
info@ofa.de  
www.ofa.de

ofa austria

**Ofa Austria**  
Wasserfeldstr. 15  
A – 5020 Salzburg  
Tel. + 43 662 848707  
Fax + 43 662 849514  
info@ofaaustria.at  
www.ofaaustria.at

ofa bamberg

dynamics®

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How to take measurements

Measurement with knee slightly bent

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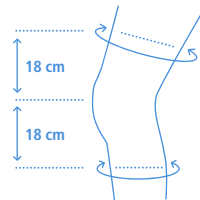
---

---

EN english	<b>Knee orthosis</b>	Adjustment	
		by experts	6
		Important notes	12
		Warranty	22

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### **Ofa Bamberg.** **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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## Adjusting the knee orthosis

(by healthcare professionals only)

The range of motion of your knee orthosis is individually adjusted by trained experts. For this purpose, stops are provided for restricting extension and flexion, together with screws and an Allen key. The stops are in pairs and are labelled with their respective number of degrees, and holes E and F are marked at the joints of the orthosis. The required extension stops (10°, 15° or 25°) are screwed to the outside of the joints at hole E (→ see [figure A, page 6](#)), and the flexion stops (40°, 60°, 75° or 90°) are screwed to the inside of the joints at hole F (→ see [figure B, page 6](#)). Both joints must have the same settings.

If the joints are not flush at the side of the leg, the provided condyle padding can be attached on the inside. To do so the cover must be removed and padding with the required thickness attached to the joint using Velcro. Then the cover is fitted again.

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---

### Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

---

**The Dynamics knee orthosis** made from lightweight, robust carbon material stabilises the knee after injuries or operations. It provides the joint with a defined range of motion that can be individually adjusted. The four-axis joint provides optimum simulation of natural knee movement.

### Fitting instructions

---

**The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.**

1

---

To put on the knee orthosis, open the Velcro tapes at the rubberised end and pull them out of the eyelet. When putting the orthosis on, please note that it becomes narrower at the lower end. Put the orthosis against the front of your leg with the leg slightly bent. It is sitting correctly if the middle of the joint is positioned at the level of the lower edge of the kneecap.

2

---

Please note that the Velcro tapes must be fastened in the correct order. First fasten the top tape at the lower leg by pulling it through the corresponding eyelet and fastening it down. Then fasten the bottom tape at the thigh, then the top one. Then fasten the lower tape at the lower leg. Please ensure that to prevent pinching the product is secure but not too tight.

---

### Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. → see page 6
- If the product has to be worn in combination with other products, then please discuss this in advance with your doctor.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then contact your doctor or specialist supplier immediately.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

## Care instructions

- The product should be washed before being used for the first time.
  - Always check the orthosis for signs of wear before use. The joints should occasionally be sprayed with dry lubricant (Teflon).
  - Detach the removable padding and Velcro tapes from the splints before washing. Please take care not to pull the Velcro strips from the splint. The splint can be cleaned with a moist cloth. In the event of heavy soiling or contact with salt water, the splint should be rinsed thoroughly and air-dried.
- Then please spray dry lubricant into the joints.
- Wash the textiles by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
  - Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
  - Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

## Indications

- instability after injuries to the front cruciate ligament and/or the collateral ligaments.
- long-term treatment of torn ligaments with lasting instability.
- postoperative stabilisation after ligament surgery

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - feeling of numbness and circulatory problems in the area of application
  - problems with lymphatic drainage as well as unexplained swelling

## Material composition

Inner material:  
60 % polyamide  
40 % polyurethane  
Frame:  
carbon (cap latex-free)

## Care

- Hand wash
  - Do not bleach
  - Do not put into a dryer
  - Do not iron
  - Do not dry clean
- [Do not use softener!](#)

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---

### Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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