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dynamics®
We'll get you moving
in no time.

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Epicondylitisbandage
Epicondylitis support

Handgelenkbandage
Wrist support

Intelligente Extras
"made in Germany"

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Intelligent extras
"made in Germany"

Combining innovative functionality, quality and design – this idea was the guideline for the development of Dynamics Plus. Be it the Plus Pad or the patented Arthroflex® comfort zone – the premium supports made by Ofa Bamberg provide you with unique extras for more effectiveness and comfort. The organic honeycomb design discreetly surrounds the anatomy and has been presented with many internationally prestigious design awards.

Attributs intelligents
"made in Germany"

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Kniebandage
Knee support

Knöchelbandage
Ankle support

Intelligente extra's "made in Germany"

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Extras inteligentes "made in Germany"

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

To put on the support, open the elastic strap. Pull the wider end of the support over the forearm, not over the elbow. With tennis elbow the pad must lie on the dorsal side of the forearm, and with golfer's elbow it must lie on the ventral side. Please wear the support directly on the skin – this is the only way to obtain the desired effect.

2

The separate Velcro strap can be worn in order to obtain a better fit. Attach the narrow hook strip of the strap to the loop strip on the support. Now lead the strap around the arm and fasten it loosely. Please ensure that the product is secure but not too tight in order to avoid pinching.

The Dynamics Plus epicondylitis

support targets the causes of tennis elbow and golfer's elbow. The large Plus Pad absorbs the vibration that is caused by hand and finger movements. This prevents irritation from being transferred to the elbow, so that the inflammation of the muscle attachments can abate.

Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff.
- Wear the product directly on your skin. If it has to be worn in combination with other products, please discuss this with your doctor beforehand.
- During long periods of rest (e. g. when sleeping) the product should be removed.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

Care instructions

- The product should be washed before being used for the first time.
- Before washing, remove the pad from the support and close the Velcro fastener of the strap. The pad must not be machine washed. Please clean the pad with a damp cloth or handwash with a mild soapy solution.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Indications

- tennis elbow
(Epicondylitis humeri radialis)
- golfer's elbow
(Epicondylitis humeri ulnaris)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

Material composition

Support:
78 % polyamide
22 % elastane
Pad:
100 % polyurethane
Laminate:
polyester

Care

- Careful wash programme
- Do not bleach
- Do not put into a dryer
- Do not iron
- Do not dry clean

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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