## Handgelenkbandage Wrist Support

ofa bamberg

**Ofa Bamberg GmbH**  
Laubanger 20  
D-96052 Bamberg  
Tel. + 49 951 6047-0  
Fax + 49 951 6047-185  
info@ofa.de  
www.ofa.de

ofa austria

**Ofa Austria**  
Wasserfeldstr. 15  
A-5020 Salzburg  
Tel. + 43 662 848707  
Fax + 43 662 849514  
info@ofaaustria.at  
www.ofaaustria.at

ofa bamberg

dynamics®

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### **Ofa Bamberg.** **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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---

### Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

### Fitting instructions

**The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.**

1

To put on the support, open the elastic strap. Pull the support over the wrist in such a way that the hand and thumb opening are pointing away from your body and the aluminium splint is on the inside of your wrist.

2

Please ensure that the recess in the silicone padding is on the ulna bone of your wrist.

3

Pass the elastic strap around your wrist and fasten it. Please ensure that the product is secure but not too tight in order to prevent pinching.

---

**The Dynamics wrist support** relieves the wrist as a result of its compressive effect and additional aluminium reinforcement. The pressure can be individually adjusted using an elastic strap. The incorporated silicone pad protects the wrist and reduces swelling.

---

### Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. The aluminium splint can be moulded when cold.
- Wear the product directly on your skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- During long periods of rest (e. g. when sleeping) the product should be removed.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, please remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

## Care instructions

---

- The product should be washed before being used for the first time.
- Remove the aluminium splint before washing.
- Close all Velcro fasteners and wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for

this product but please do not wring it out.

- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments

or lotions to the skin in the usage area. These substances can irritate the material of the product.

- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

## Indications

---

- wrist irritation (chronic, after injuries or operations)
- unstable wrist
- tendon sheath inflammation (tendovaginitis)
- joint degeneration (arthrosis)

## Side effects

---

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

---

In the following cases you should consult your doctor before using this product:

- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- feeling of numbness and circulatory problems in the area of application
- problems with lymphatic drainage as well as unexplained swelling

## Material composition

---

Support:

77 % polyamide

15 % elastodiene

8% elastane

Stabilisation element:

aluminium

This product contains latex, which can lead to allergic reactions.

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---

### Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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