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Instructions for use

Knieorthese Soft Expert

Knee Orthosis Soft Expert

dynamics[®]

ofa bamberg

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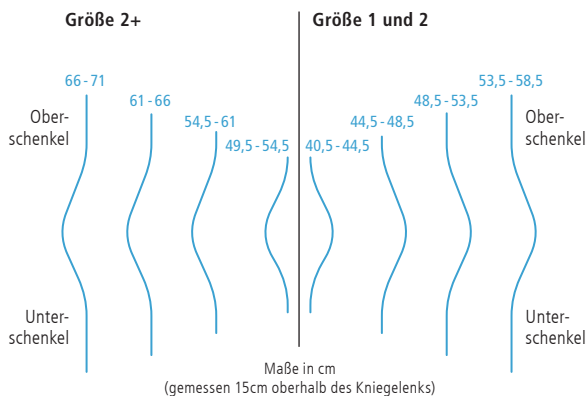
We have a lot to offer

the medical
compression
specialist
since
1928
made
in Germany

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Ofa Bamberg. **More than 90 years' experience** **with your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy. In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive support and travel stockings.

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Fitting of the Knee Orthosis

(only by experts)

The circumference of the cuff and the range of motion of your knee orthosis are tailored individually to you by trained medical/orthopaedic experts.

The joint splints and the Velcro straps need to be removed to adjust the circumference of the cuff. The two side parts can then be removed from the back of the brace and reattached depending on the circumference of your thigh. Size markers are stitched into the back of the brace to make it easier to adjust (→ see Figure on Page 3). The cuff is positioned correctly if the leg is completely enclosed but the kneecap remains free.

After adjusting the cuff, the joint splints must be inserted back into the cuff such that the joints are positioned on the side at the level of the knee.

The joint cover can be folded back to adjust the joints. The metal pins are used to adjust the flexion and extension. (→ see Figure A, Page 2).

These must be set to the same level on both axes of a joint. Both of the joints must also have the same angle settings.

The joint cover is closed again once the adjustment is complete.

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Introduction

Dear user, we are delighted that you have chosen an Ofa Bamberg product and we want to thank you for putting your trust in us. Please take a moment to read this product information through carefully to ensure your Dynamics product works as effectively as possible.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

1

To put the brace on, open all of the Velcro fasteners, pull the Velcro straps out of the eyelets and fold back the cuff fully. When putting the cuff on, please be aware that it gets narrower at the lower end. Place the back of the brace against the leg and close the cuff on the lower leg and then the thigh. The brace is correctly positioned if the kneecap is in the corresponding opening.

2

Ensure that you close the Velcro straps in the right order. Close the upper strap on the lower leg by inserting it through the eyelet provided and fastening it tightly. Then close the lower strap on the thigh and then the lower one on the lower leg. Then close the upper strap on the thigh. Ensure that the product is secure but not too tight so as to prevent pinching.

Intended Use

The Dynamics Knee Orthosis Soft Expert is designed to stabilise and immobilise your knee joint in part or in full. Aluminium stabiliser bars and an adjustable joint hinge limit the range of movement of your knee joint, helping to support its healing process, for example after an operation.

Important information

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists. [See page 5.](#)
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

english

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Remove the joint splints before washing. Open up the joint cover and note the angle so you can set it back to this level after cleaning. Now remove the metal pins in a "flexion" position so the splints can be folded up and pulled out. The splints can be cleaned with a damp cloth.
- Close all of the Velcro fasteners and wash the product by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.

- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Hand wash
 -  Do not bleach
 -  Do not tumble dry
 -  Do not iron
 -  Do not dry clean
- Do not use fabric softener!**

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

- Postoperative after ligament sutures or ligament reconstructions
- Mobilisation after stabilised fractures close to the joint

Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

Material composition

46 % Nylon
 20 % Aluminium
 11 % Polyester
 9 % PU foam
 7 % Polypropylene
 3 % Polyoxymethylene
 2 % ABS
 1.4 % Steel
 0.5 % Elastane
 0.1 % Rubber

This product contains Latex, which can cause allergic reactions.

Disposal

Please dispose of the product according to local regulations after the end of use.

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Garantie

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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Ofa Bamberg GmbH