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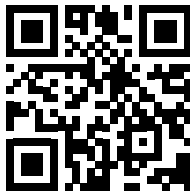
Instructions for use

ofa[®]

Produktinformationen und Gebrauchsanweisung - online
Product information and instructions for use - online

dynamics[®] plus

Knöchelbandage

REF 44 7112

EN Ankle Support

NL Enkelbandage

IT Cavigliera

SV Ankelstöd

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englisch

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

- 1** The ankle support can be put on quickly and easily using the enclosed fitting aid. This is done by slipping your foot into the opening of the fitting aid.
- 2** Now pull the support over the foot with the longitudinal seam facing the rear. The support is sitting correctly if the silicone pad is surrounding the ankle from below.
- 3** Finally, remove the fitting aid by pulling it out of the support over the toes towards the front.

Purpose

The product is used to compress the ankle joint. The compressive effect stabilises the ankle joint. Silicone pads around the inner and outer ankle support the reduction of swelling.

Important notes

- The initial fitting and positioning of the product as well as training in its proper, safe use should be carried out by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and

damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.

- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before first use and regularly thereafter.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

- 🧺 Delicate wash cycle
- ⚠ Do not bleach
- 🚫 Do not tumble dry
- 🚫 Do not iron

🚫 Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- Tendinitis
- Aftercare following damage to the ankle and the ligaments, e. G. Contusions, sprains, capsule lacerations, collateral ligament ruptures (fibular ligament rupture)
- Joint wear (arthrosis), joint inflammation (arthritis)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using this product:

- Skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- Feeling of numbness and circulatory problems in the area of application
- Problems with lymphatic drainage as well as unexplained swelling

Material composition

44% Polyamide
44% Silicone
7% Elastane
3% Viscose
2% Polyester
Aluminum

Disposal

Please dispose of the product according to local regulations after the end of use.

Guarantee

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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Ofa Bamberg GmbH