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## Instructions for use

---

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### Bandage Shoe

---

dynamics<sup>®</sup>

ofa bamberg

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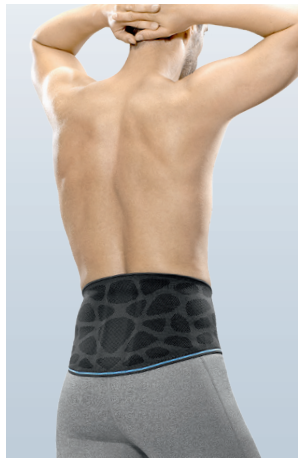
**Orthopädie**  
Orthopaedics

since  
**1928**  
the medical  
compression  
specialist  
made  
in Germany

**Ofa Bamberg.**  
**More than 90 years' experience**  
**with your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy. In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive support and travel stockings.

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**Intended Use**

---

**The product** is designed to relieve the strain on your foot. The stiff outsole prevents too much pressure on your foot, helping to support the healing process, for example after an operation.

---

**Fitting instructions**

---

**The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.**

**1**

If the shoe is worn with a toe cap, attach this before putting the shoe on by lifting the insole in the toe area and fastening the toe cap underneath.

**2**

To put the shoe on, open all of the Velcro fasteners and tilt the toe cap forwards. Put your foot in the shoe and close the Velcro fasteners over your toes and instep.

**3**

Then fold the toe cap back and fasten it to the side of the shoe.

**4**

Close the Velcro strap at the ankle in such a way that the tongue is underneath the toe cap. Pull the Velcro strap through the eyelet and fasten it tightly. Ensure that the product is secure but not too tight so as to prevent pinching.

---

 **Important information**

---

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

# english

## Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Close all of the Velcro fasteners and wash the product by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

## Care

-  Hand wash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

## Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

## Disposal

Please dispose of the product according to local regulations after the end of use.

## Indications

- After operations and injuries in the forefoot area
- Rheumatoid foot
- Wounds or diseases in the forefoot area (suitable for diabetics)

## Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves. There can be local temperature increases. If this happens, briefly elevate the foot and open up the product to avoid heat accumulating. For patients with polyneuropathy (nerve disorder) in the feet, it is recommended that the bandage be checked daily.

## Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling

- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

## Material composition

- 43 % EVA
- 23 % Polyester
- 15 % Polyamide
- 6 % Foam
- 5 % Rubber
- 5 % Cardboard
- 2 % POM
- 1 % PU

This product contains Latex, which can cause allergic reactions.

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#### **Garantie**

---

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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### **Ofa Bamberg GmbH**