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Instructions for use

Achillesbandage

Achilles Tendon Support

Plus-Pelotte

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Arthroflex® Komfortzone

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ofa bamberg

dynamics®
plus

Anziehanleitung in Bildern

Fitting instructions
in pictures

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dynamics[®] plus
Lumbalbandage
Lumbar Support

dynamics[®] plus
Handgelenkbandage
Wrist Support

dynamics[®] plus
Epicondylitisbandage
Epicondylitis Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex[®] comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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dynamics[®] plus
Ellenbogenbandage
 Elbow Support

dynamics[®] plus
Kniebandage
 Knee Support

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Introduction

Dear user,

Please take a moment to read through this product information carefully to ensure optimal effectiveness and safe use of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

The support can be put on quickly and easily using the enclosed donning aid. This is done by slipping your foot into the opening of the donning aid.

2

Now put the support on your foot, the long seam towards the back. The support is positioned correctly if the silicone pad is centred over your Achilles tendon and the Arthroflex zone is at the front of the ankle.

3

Lastly remove the donning aid, pulling it forward out of the support and over your toes.

4

To remove, insert both thumbs into the top edge and pull the support off over the heel.

Purpose

The compression effect of the Dynamics Plus Achilles Tendon Support relieves the area around the Achilles tendon. The massaging silicone truss pad helps to quickly reduce swelling. The extra thin, extremely stretchy Arthroflex® comfort zone is built into the sensitive area at the front of the ankle for comfort.

Important notes

- The initial fitting and adjustment of the product should be carried out and instructions on its proper and safe use provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Care instructions

- The product should be washed before its first use and then on a regular basis.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right

shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.

- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the

material of the product.

- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Instructions for care

-  Delicate wash cycle
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- Chronic or post-traumatic irritation in the Achilles heel area (e. g. achillodynia)
- Irritation with Haglund's deformity
- Post-operative (e. g. aftertreatment after a ruptured Achilles heel)
- Mucous membrane inflammations (e. g. bursitis subachillea)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- Skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - Feeling of numbness and circulatory problems in the area of application
 - Problems with lymphatic drainage as well as unexplained swelling

Material composition

42 % silicone
41 % polyamide
6 % elastane
3 % viscose
3 % polyester
3 % cotton
2 % polyurethane
aluminium

Disposal

Please dispose of the product according to local regulations after the end of use.

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Garantie

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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Ofa Bamberg GmbH