PUSH MED SHOULDER BRACE PLUS

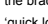
INTRODUCTION

Push med products provide effective solutions for the treatment and prevention of injuries of the locomotor apparatus.

Our new shoulder brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med shoulder brace Plus ensures proper fixation and pain relief. Mechanical support and stable shoulder positioning constitute the basis of its performance.

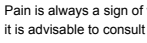
Push med products are innovative and are manufactured for you with great care. We are committed to providing you maximum confidence in the interest of satisfied users. Read the product information carefully before you use the Push med shoulder brace Plus. If you have any questions, do not hesitate to ask your supplier or use www.push.eu (info@push.eu)

PRODUCT CHARACTERISTICS

Your Push med shoulder brace Plus consists of four components. The breast strap, which is closed around the chest or upper waist, constitutes the basis of the brace. This strap only needs to be adjusted once for the correct size. The 'quick lock' () can be used to open and close the breast strap and to adjust it for maximum comfort. To prevent abduction of the shoulder as much as possible, the upper arm strap fixes the upper arm against the body. The comfortable wrist hand support keeps the arm fixed and supports it. The Push med shoulder brace Plus adds comfort through the shoulder strap that has been added to the brace: this strap provides additional immobilisation and support to the affected shoulder joint and allows the hand to be kept at the desired height. Each fixation point is individually adjustable, for an optimal fit in all circumstances. The brace is made of Sympress™, a comfortable and high-quality material. The microfibre used makes the inside soft to the touch and keeps the skin dry. The edges of the straps also feel soft. The Push med shoulder brace Plus can be washed normally.

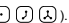
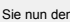
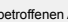
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
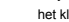
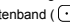
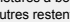
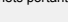
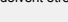
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
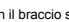
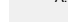
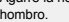
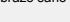
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
INDICATIONS

- Collarbone fracture
 - Treatment after (sub)luxation
 - Treatment after (sub)luxation of the SC or AC joint
 - Postoperative treatment
 - Humeroscapular periarthritis
 - Subacromial humerus fracture
 - As a sling in the case of a facciali hemiparesis (also at night)
- There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the brace too lightly in the beginning.
- Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

FITTING / SETTING

The shoulder brace is available in 3 sizes. For proper fitting, the chest circumference is measured 5 centimetres above the level of the hanging elbow. The table on the packaging shows the correct brace size for your chest circumference.

The correct fitting sequence is determined by the increasing molecule symbols on the hook and loop fasteners (, , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , , ,

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