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dynamics®  
We'll get you moving  
in no time.

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### **Ofa Bamberg.** **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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---

**Dear user,**

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

---

**Fitting instructions**

**The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.**

---

**1**

To put on the orthosis, open the Velcro fastener of the lacing. Pull the orthosis over the wrist in such a way that the hand and thumb opening is pointing away from your body and the moulded aluminium splint is on the inside of your wrist.

---

**2**

Now close the orthosis by guiding the lacing around your wrist and securing it by Velcro fastening it on the top of the orthosis. Please ensure that the product is secure but not too tight in order to prevent pinching.

---

**3**

Finally, adjust, if necessary, the Velcro fastener between your thumb and your index finger.

---

**The Dynamics lace-up wrist brace**

provides optimum wrist immobilisation. An anatomically contoured aluminium splint positions the wrist in a relaxing neutral position. The practical lace-up system makes the orthosis easy to put on and distributes the pressure evenly.

---

**Please note**

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. The aluminium splints can be moulded when cold.
- Wear the product directly on your skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

# english

## Care instructions

- The product should be washed before being used for the first time.
- Remove the aluminium splints before washing.
- Wash the product by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

## Indications

- wrist irritation (chronic, after injuries or operations)
- unstable wrist
- tendon sheath inflammation (tendovaginitis)
- minor carpal tunnel syndrome (nerve damage in the hand area)

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - feeling of numbness and circulatory problems in the area of application
  - problems with lymphatic drainage as well as unexplained swelling

## Material composition

- 35 % Polyester
- 30 % PVC
- 25 % PU foam
- 5 % Polyamide
- 5 % Aluminium

## Care

-  Hand wash
  -  Do not bleach
  -  Do not put into a dryer
  -  Do not iron
  -  Do not dry clean
- Do not use softener!**

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---

### Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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