## PUSH ORTHO REHABILITATION SOCKS

### INTRODUCTION

Push ortho products provide effective solutions in the case of disorder or reduced functioning of the locomotor apparatus and are recommended by doctors and scientists. Your new rehabilitation stocking is characterised by high effectiveness, comfort and quality. Before using this product, please read the product information carefully and follow the fitting instructions. If you have any questions please contact your doctor or specialist dealer, or send an e-mail to kontakt@ofa.de.

### INTENDED USE

The Push ortho rehabilitation sock is a medical knee-high compression stocking, especially designed for the prevention and/or therapy of post-operative and post-traumatic oedema and swelling of the lower extremities.

### EFFECTIVE PRINCIPLE OF MEDICAL COMPRESSION STOCKINGS

Products tailored to the respective indication and controlled pressure accelerate blood reflux, improve microcirculation in the tissue and increase lymph flow.

### ⚠ IMPORTANT INFORMATION

- Only wear compression stockings after consultation with your doctor.
- Wearing your stockings daily and holding regular consultations with your doctor will ensure successful treatment! The stockings must be obtained from a medically trained specialist retailer. Your specialist retailer will also advise you on how to put the compression stockings on correctly.
- If you experience leg pain or skin irritation while wearing these stockings, please consult your doctor immediately.

- Please contact your doctor about wearing the stockings in combination with using other medical devices.
- We test our products within the scope of a comprehensive quality assurance. However, if you still have cause for complaint, please contact your specialist retailer.
- Do not attempt to perform repairs yourself. Return stockings to the specialist retailer who sold them to you.
- Any pilling of the material that may arise due to friction with other garments will not reduce the stocking's quality.

### INDICATIONS

- Prevention and treatment of post-operative and post-traumatic oedema
- Medical compression therapy

### ABSOLUTE CONTRAINDICATIONS

- Advanced peripheral arterial occlusive disease (if any of the following values apply: ABPI < 0.5, ankle arterial pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO<sub>2</sub> < 20 mmHg on instep)
- Congestive heart failure
- Septic phlebitis
- Phlegmasia cerulea dolens

### RISKS TO BE CONSIDERED (THERAPY DECISION TO BE MADE BASED ON BENEFIT/RISK ASSESSMENT)

- Weeping dermatoses
- Allergies to compression stocking materials
- Severe sensitivity disorders of the extremity
- Advanced peripheral neuropathy (e.g. diabetes mellitus)
- Chronic polyarthritis

**No liability is accepted if these contraindications are not observed.**

### RISKS AND SIDE EFFECTS

Medical compression stockings can cause skin necrosis and pressure damage to the peripheral nerves. Patients with sensitive skin may experience

itching and skin irritation when using compression stockings. Adequate skin care is therefore recommended. If you suffer the following symptoms please take off the stockings immediately and discuss further treatment with your doctor: blue or white coloration of the toes, sensations of discomfort and numbness, increasing pain, shortness of breath and sweating, acute movement restrictions.

### HOW TO PUT THEM ON

1. To put the stockings on, turn it inside out as far as the foot. Slip the foot section over your toes and then over the instep.
2. Now carefully pull the leg section over your heel. Please avoid pulling up the stockings at the end or grip top.
3. Pull the rest of the stockings up evenly without overstretching it. Finally, adjust the stockings on the leg until there are no wrinkles.

### TIPS FOR DONNING THE STOCKINGS

We recommend wearing household rubber gloves to make it easier to put on the stockings and to protect the knitted fabric. Jewellery and long fingernails can damage the stockings. Please also pay attention to regular foot care and proper footwear. Ask your medical retailer about Ofa Grip special gloves and Ofa Fit Magnide, the practical donning and doffing aid for compression stockings from Ofa Bamberg, available with open and closed toe. Ofa Fit Magnide is made of very smooth canvas material and enables the stockings to slide easily over the foot.

### CARE INSTRUCTIONS

**Please wash inside out and do not use fabric softener!**

-  Wash on a delicate or gentle cycle
-  Do not bleach
-  Dry gently and carefully
-  Do not iron
-  No chemical dry cleaning

The product is suitable for immediate use. We recommend washing the product before wearing it for the first time. Then wash daily by hand or on a delicate fabric program. Wash the stockings separately with mild detergent without fabric softener (e.g. Ofa Clean special detergent) at a maximum temperature of 40 °C and rinse them well. The stockings can be spun gently, but do not wring them out. Dry your compression stockings in air or in a dryer on a gentle cycle. Do not place them on a heater or in direct sunlight. Never iron the stockings. To maintain the effectiveness of your stockings, please do not use chemical cleaning agents, petrol, or fabric softeners. These substances damage the knitted fabric. Compression stockings are largely resistant to fats and oils, but should not come into contact with ointments. Store the stockings in a dry place away from sun and heat, in the original packaging if possible.

### WARRANTY

The shelf life of medical compression stockings is limited to 3 years when properly stored. Added to this is the wearing time, which is a maximum of 6 months with regular use and proper care. After this period, the products are excluded from the warranty. Defects attributable to normal wear and tear or improper use are not covered by the guarantee. Improper use also includes any subsequent modification to the product that has not been carried out by the manufacturer. The product is intended for use by a single patient. If it is passed on to other persons for reuse, the manufacturer's product liability is no longer valid.

### DISPOSAL

After the end of the usage period, please dispose of the product in accordance with local regulations.

### MANDATORY REPORTING

Due to legal requirements, patients and users are obliged to report immediately any serious incidents which have occurred in connection with the product and which have resulted in, or could have led to a significant deterioration in the state of health or to death to the manufacturer and the competent national authority (in Germany BfArM).

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Ofa Austria  
Wasserfeldstr. 20  
A – 5020 Salzburg  
www.ofaustria.at

Designed and made  
in Germany

Oeko-TEX®  
CONFIDENCE IN TEXTILES  
STANDARD 100  
95.0.1594 HOHENSTEIN HTTI

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