LWS Bandage Lumbar Spine Support

ofa bamberg

Ofa Bamberg GmbH
Laubanger 20
D-96052 Bamberg
Tel. +49 951 6047-0
Fax +49 951 6047-185
info@ofa.de
www.ofa.de

ofa austria

Ofa Austria
Wasserfeldstr. 15
A-5020 Salzburg
Tel. +43 662 848707
Fax +43 662 849514
info@ofaaustria.at
www.ofaaustria.at

dynamics®

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dynamics®
We'll get you moving
in no time.

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Ofa Bamberg. **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

Before putting on the support, please open the Velcro fastener and check the position of the pad. Its tip should be pointing downwards, and can be Velcro fastened to the desired height on the inside of the support.

2

The support comes with loops to make it easier to put on. First lead your left hand through the inner loop and place the support around your body.

3

Then lead your right hand through the outer loop and close the Velcro fastener in front of your stomach.

The Dynamics lumbar spine support

stabilises and provides compressive relief to the lumbar region. A height adjustable pad massages the muscles with soft nubs. This stimulates blood circulation and helps to relieve tension in the lumbar spine area.

Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff.
- Wear the product over a thin close-fitting shirt. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- During long periods of rest (e. g. when sleeping) the product should be removed.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

english

Care instructions

- The product should be washed before being used for the first time.
- Remove the pad from its sleeve before washing. It can be cleaned with a moist cloth. To insert the pad correctly after cleaning, the nubbed side must face downwards when placed in the sleeve (figure 4).
- Close all Velcro fasteners and wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild

detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well.

A gentle spin can be used for this product but please do not wring it out.

- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning

products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.

- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Indications

- acute and chronic pain in the lumbar and sacral areas of the spine
- lumbago
- degenerative changes to the lumbar spine

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using this product:

- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- feeling of numbness and circulatory problems in the area of application
- problems with lymphatic drainage as well as unexplained swelling

Material composition

55 % Polyamide
39 % Elastodiene
6 % Elastane

This product contains latex, which can lead to allergic reactions.

Care

-  Careful wash programme
-  Do not bleach
-  Do not put into a dryer
-  Do not iron
-  Do not dry clean

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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