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Instructions for use

Fersenkissen

Heel Cushion

dynamics®

ofa bamberg

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Orthopaedics

Ofa Bamberg.

**More than 90 years' experience with your
health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy.

In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive travel and support stockings.

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Introduction

Dear user, we are delighted that you have chosen an Ofa Bamberg product and we want to thank you for putting your trust in us. Please take a moment to read this product information through carefully to ensure your Dynamics product works as effectively as possible.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

1

The heel cushions are placed in your shoes. Please note the "rechts" (right) and "links" (left) markings.

Intended Use

The Dynamics Heel Cushions are designed to relieve strain on the heel. They are made of viscoelastic material, which relieves strain on the heel and the joints above it (the ankle, knee and hip joints), as well as on the lumbar spine. This facilitates healing processes.

Important information

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product is fitted correctly. Check the product to ensure it is working and check it for wear and damage (such as breaks, cracks or distortion) before each use. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

english

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Wash the product by hand using a mild detergent. It can be reused immediately once it has been dried with a towel.
- Do not dry the product on a radiator or in direct sunlight.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Hand wash
-  Do not bleach
-  Do not put into a dryer
-  Do not iron
-  Do not dry clean

Do not use softener!

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Side effects

There are no known side effects if the product is used properly.

Indications

- Heel spur
- Inflammation of the Achilles tendon (achillodynia)
- Ankle, knee and hip degeneration (arthrosis)
- Degenerative changes to the spine

Contraindications

- Consult your doctor before using this product in the following cases:
- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
 - Feeling of numbness and circulatory problems in the area where the product is used
 - Problems with lymphatic drainage and unexplained swelling

Material composition

100 % pure Silicone

Disposal

Please dispose of the product according to local regulations after the end of use.

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Garantie

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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Ofa Bamberg GmbH