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Instructions for use

### Knöchelbandage

Ankle Support

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dynamics® plus  
**Lumbalbandage**  
Lumbar Support

dynamics® plus  
**Handgelenkbandage**  
Wrist Support

dynamics® plus  
**Epicondylitisbandage**  
Epicondylitis Support

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### The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex® comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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---

## Introduction

**Dear user,** we are delighted that you have chosen an Ofa Bamberg product and we want to thank you for putting your trust in us. Please take a moment to read this product information through carefully to ensure your Dynamics Plus product works as effectively as possible.

**Ofa Bamberg wishes you a speedy recovery!**

---

## Fitting instructions

**The fitting instructions in pictures can be found in the front cover (2) of the brochure.**

---

### 1

The ankle support can be put on quickly and easily using the enclosed fitting aid. This is done by slipping your foot into the opening of the fitting aid.

---

### 2

Now pull the support over the foot with the longitudinal seam facing the rear. The support is sitting correctly if the silicone pad is surrounding the ankle from below.

---

### 3

Finally, remove the fitting aid by pulling it out of the support over the toes towards the front.

---

## Purpose

**The Dynamics Plus Ankle Support** stabilises the ankle joint by means of its compressive effect. Massaging silicone pads help reduce swelling rapidly. For a pleasant wearing feeling the extremely thin, highly elastic Arthroflex® comfort zone is incorporated in the sensitive area of the talus.

---

## Important information

- The initial fitting and adjustment of the product should be carried out and instructions on its proper and safe use provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

## Instructions for care

- The product should be washed before first use and regularly thereafter.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right

shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.

- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the

material of the product.

- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

## Care

- Delicate wash cycle
- Do not bleach
- Do not tumble dry
- Do not iron
- Do not dry clean

**Do not use fabric softener!**

## Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

## Indications

- Tendinitis
- Aftercare following damage to the ankle and the ligaments, e. G. Contusions, sprains, capsule lacerations, collateral ligament ruptures (fibular ligament rupture)
- Joint wear (arthrosis), joint inflammation (arthritis)

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

- In the following cases you should consult your doctor before using this product:
- Skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - Feeling of numbness and circulatory problems in the area of application
  - Problems with lymphatic drainage as well as unexplained swelling

## Material composition

44% Polyamide  
44% Silicone  
7% Elastane  
3% Viscose  
2% Polyester  
Aluminum

## Disposal

Please dispose of the product according to local regulations after the end of use.

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---

### **Garantie**

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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### **Ofa Bamberg GmbH**