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## Instructions for use

---

### ROM Knieschiene

### ROM Knee Splint

---

dynamics<sup>®</sup>

ofa bamberg

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---

---

<b>EN</b> english	<b>ROM Knee Splint</b>	Fitting by experts	6
		Important information	12
		Guarantee	22

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### **Ofa Bamberg.** **More than 90 years' experience** **with your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy.

In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive travel and support stockings.

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(only by experts)

---

The ROM Knee Splint is individually tailored to you by trained experts. The length of the lateral splints can be adjusted to any one of five levels at the thigh and lower leg. This is done by pressing and holding the oval, grey button and the splint is shortened or extended as required.

→ See Figure A, page 6

The transparent cover can be rotated and opened to adjust the ROM joints by pushing the cap outwards and flipping it open. The metal pins are used to adjust the flexion and extension. Both joints should always be set to the same levels. The cover is closed again once the adjustment is complete.

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---

**Introduction**

**Dear user**, we are delighted that you have chosen an Ofa Bamberg product and we want to thank you for putting your trust in us. Please take a moment to read this product information through carefully to ensure your Dynamics product works as effectively as possible.

**Ofa Bamberg wishes you a speedy recovery!**

---

**Fitting instructions**

**The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.**

**1**

To put the product on, open all of the Velcro fasteners and the fabric cuff. When putting the product on, please be aware that the cuff gets narrower at the lower end. Place the splint against the back of the leg so that the ROM joints are positioned in line with the middle of the knee.

**2**

First close the wide cuff on the lower leg, then the one on the thigh. Then close the two narrow cuffs.

**3**

Then close the Velcro straps in the same order by inserting them through the eyelet provided and fastening them tightly. If necessary you can then adjust the cuff and the Velcro fasteners until the splint is correctly positioned. Ensure that the product is secure but not too tight so as to prevent pinching.

---

**Intended Use**

**The Dynamics ROM Knee Splint** is designed to stabilise and immobilise your knee joint in part or in full. Metal stabiliser bars and an adjustable joint hinge limit the range of movement of your knee joint, helping to support its healing process, for example after an operation.

---

 **Important information**

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

## Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Detach the removable padding from the splints for washing. Ensure that you do not pull the Velcro strips off the splints too. The splints can be cleaned with a damp cloth.
- Wash the fabric by hand. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

## Care

-  Hand wash
  -  Do not bleach
  -  Do not tumble dry
  -  Do not iron
  -  Do not dry clean
- Do not use fabric softener!

## Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

## Indications

- After injuries or operations to immobilise the knee joint with subsequent mobilisation in defined ranges of motion

## Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

## Material composition

45 % Nylon  
21 % Aluminium  
15 % Steel  
8 % PU foam  
6 % Polyester  
3 % Cotton  
2 % POM  
1 % Silicone

## Disposal

Please dispose of the product according to local regulations after the end of use.

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---

### **Garantie**

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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### **Ofa Bamberg GmbH**