## Verbandschuh Multivario Bandage Shoe Multivario

ofa bamberg

**Ofa Bamberg GmbH**  
Laubanger 20  
D-96052 Bamberg  
Tel. +49 951 6047-0  
Fax +49 951 6047-185  
info@ofa.de  
www.ofa.de

ofa austria

**Ofa Austria**  
Wasserfeldstr. 15  
A-5020 Salzburg  
Tel. +43 662 848707  
Fax +43 662 849514  
info@ofaaustria.at  
www.ofaaustria.at

ofa bamberg

dynamics®

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**Orthopädie**  
Orthopaedics

the medical  
compression  
specialist  
since  
1928  
made  
in Germany

**Ofa Bamberg.**  
**More than 85 years' experience**  
**for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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(by experts only)

---

The Multivario insole is adjusted individually for you by trained experts. For targeted pressure relief (e. g. in the vicinity of a wound), individual honeycombs can be removed from the sole. If necessary the entire sole can be removed (attached using Velcro) and replaced with a customised insole.

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EVA

**Dear user,**

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

**Fitting instructions**

---

**The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.**

**1**

To put on the shoe, open all Velcro fasteners. The toe cap can be removed if necessary by lifting the insole at the front of the shoe and releasing the toe cap.

**2**

Put your foot in the shoe and close the Velcro fastener over the instep. Then close the Velcro fastener at the ankle by pulling it through the provided eyelet and then fastening it. Please ensure that the product is secure but not too tight in order to prevent pinching.

**The Dynamics bandage shoe**

**Multivario** provides targeted relief of the foot in the vicinity of wounds or in the event of pressure pain. This is done using a special insole with removable honeycomb elements. This extremely light shoe can be worn with the toe open or closed.

**Please note**

---

- The initial adjustment and strapping on of the product must be carried out by trained experts. It might be necessary to make adjustments for uneven leg lengths.  
→ see page 6
- Do not wear the products directly on injured skin. Daily bandage checking is recommended for patients with polyneuropathy (nerve disease) in the feet.
- Do not drive a car or any other vehicle whilst wearing the shoe.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, contact your doctor or specialist supplier immediately.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

## Care instructions

- Wash the product by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.

## Indications

- for diabetics with severe circulation problems and ulcer formation (ulceration)
- for rapid mobilisation after foot operations
- for protection and relief in the event of pressure pain, e. g. caused by bursa inflammation (Bursitis) and hammer toe (Hallux Valgus)

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. These substances can affect the material of the product.

## Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - feeling of numbness and circulatory problems in the area of application
  - problems with lymphatic drainage as well as unexplained swelling

## Care

- 👑 Hand wash
  - ⊗ Do not bleach
  - ⊗ Do not put into a dryer
  - ⊗ Do not iron
  - ⊗ Do not dry clean
- Do not use softener!

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#### **Warranty**

---

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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