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instructions for orthopaedic specialist

Inhalt
Contents
Sommaire
Inhoud
Indice
Índice

DE deutsch	Lumbalbandage	Wichtige Hinweise Gewährleistung	6 18
EN english	Lumbar Support	Important notes Guarantee	8 18
FR français	Bandage lombaire	Remarques importantes Garantie	10 18
NL nederlands	Lumbaalbandage	Belangrijke informatie Garantie	12 18
IT italiano	Fascia lombare	Avvertenze importanti Garanzia	14 18
ES español	Faja lumbar	Advertencias importantes Garantía	16 18

dynamics[®] plus
Handgelenkbandage
Wrist Support

dynamics[®] plus
Epicondylitisbandage
Epicondylitis Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex[®] comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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dynamics® plus
Kniebandage
 Knee Support

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57% Polyamid
18% Elasthan
16% Polyurethan
9% Metall
Polypropylen
Acrylamid
Polyester
Venyl
Aluminium

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Introduction

Dear user, we are delighted that you have chosen an Ofa Bamberg product and we want to thank you for putting your trust in us. Please take a moment to read this product information through carefully to ensure your Dynamics Plus product works as effectively as possible.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

Before you put on the support, open the Velcro closure and check the position of the pad. It should be placed with the tip facing downward and can be Velcroed to the desired height on the inside of the support.

2

The support has a hand loop that makes it easier to put on and take off the support. The loop can be adjusted to the individual needs of right- or left-handed users so that their dominant hand opens and closes the support. First put your preferred hand through the loop, then place the support around your body.

3

Then put your second hand into the semicircular strap at the other end of the support and close the product in front of your stomach.

4

The support is fitted correctly when the two Arthroflex® zones are positioned at the level of your hip bones.

Purpose

The Dynamics Plus Lumbar Support helps you when you have pain in your lumbar spine. The product promotes more upright gait, helping to alleviate pain. The Plus Pad, is a massaging pad has soft knobs to stimulate circulation around the lumbar spine. Two thin, highly elastic Arthroflex® comfort zones in the pelvic bone region provide additional wearer comfort.

Important information

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists. → see page 7.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product over a thin, close-fitting shirt. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Remove the pad from the support before washing. It can be cleaned with a damp cloth.
- Close all Velcro closures and wash the product separately in a washing bag at not more than 30°C. Use mild-acting detergents without plasticisers (such as Ofa Clean special detergent) and rinse the product out thoroughly. The product can be spun gently, but please do not wring it out.
- Mould the product into the right shape and let it air-dry. Do not dry the product in the dryer, on a radiator/heater or in direct sunlight, and never iron it.
- In order to maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric conditioners. Do not apply any greasy or acidic products, ointments or lotions to the skin in the area of use. These substances

may damage the material.

- Store the product in a dry place and protected from sunlight and heat, preferably in the original packaging. Please dispose of the product according to local specifications after the end of use.

Disposal

Please dispose of the product according to local regulations after the end of use.

Indications

- Acute and chronic pain in the lumbar and sacral regions of the spine
- Lumbago
- Degenerative changes to the lumbar spine

- Feelings of numbness and circulatory problems in the area of application
- Problems with lymphatic drainage as well as unexplained swelling

Material composition

57% Polyamide
 18% Elastane
 16% Polyurethane
 9% Metal
 Polypropylene
 Acrylamide
 Polyester
 Vinyl
 Aluminum

Adjusting the lumbar support (for specialist staff only)

The closure of your Dynamics Plus Lumbar Support will be adjusted to your individual needs by specially trained staff. [→ see page 2] **A** The product features a neutral angle setting for right-handed wearers.

B The two parts of the closure can be detached from the support to adjust them to the shape of your body. To put them back on, the adjusting aid provided is placed between the Velcro and the fleece. The desired angle can now be set, the adjusting aid removed and the Velcro pressed closed. **C** to fit the closure for left-handed wearers, the two parts of the closure are detached from the support and each is reattached to the other side, as described in step **B**. **D** The product is positioned correctly if both parts of the closure are fastened together and set horizontally flush on the stomach.

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this may cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using the product:

- Skin conditions or injuries in the area of application, particularly where signs of inflammation such as redness, heating or swelling are present

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Garantie

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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Ofa Bamberg GmbH