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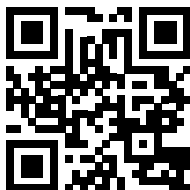
Instructions for use

ofa[®]

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english

Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

- 1** To put the orthosis on, open the two wide Velcro straps and place your foot on the heel part. Place the shells against the ankle and leg and check the settings on the heel part. If this has been adjusted correctly, the heel part and shells will be flush but not too tight.
- 2** If necessary, correct the width of the heel part using the Velcro fasteners on the shells.
- 3** Hold the orthosis against your leg with one hand. First wrap the lower Velcro strap tightly around the shells and fasten it, then fasten the upper Velcro strap in the same way. You can then readjust the straps if necessary. Ensure that the product is secure but not too tight so as to prevent pinching.

Intended Use

The product is designed to stabilise and partially immobilise the ankle joint. Two lateral stabilisation elements made of solid plastic limit the degrees of freedom of the ankle joint.

Important information

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Detach the removable padding and the heel part from the shells for washing. Ensure that you do not pull the Velcro strips off the shells too. The shells can be cleaned with a damp cloth.
- Wash the fabric parts by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well. Squeeze the water out carefully without wringing the product.
- Mould the product into the right shape and let it

dry in the air.

- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Handwash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

- Damage to the ankle and the ligaments, e.g. bruising, sprains, capsule lacerations
- For stabilisation after ligament sutures

Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an

- increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

Material composition

- 30% Polyester
- 29% PU Foam
- 25% Polypropylene
- 15% Nylon
- 1% Steel

Disposal

Please dispose of the product according to local regulations after the end of use.

Garantie

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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Ofa Bamberg GmbH