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## Instructions for use

---

### Hallux Valgus Nachtschiene

#### Hallux Valgus Night Splint

---

dynamics®

ofa bamberg

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### ← Fitting instructions

---

**The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.**

1

---

Place your big toe in the front part of the splint so that the wide leather strap lies against the outside of the ball of your foot.

2

---

To secure the splint, guide the narrow Velcro strap around the outside of your foot and through the notch in the splint on the top of your foot. Tighten it until the required correction position is reached, and fasten the strap. Ensure that the product is secure but not so tight that it causes pinching.

---

### Introduction / Intended use

---

#### **Dear user,**

The Hallux Valgus Night Splint from Dynamics uses an individually adjustable lever mechanism to correct the position of your metatarsophalangeal joint while you wear it. The soft, skin-friendly inner padding guarantees exceptional comfort.

**Ofa Bamberg wishes you a speedy recovery!**

---

** Important information**

---

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

---

## Important information

# Hallux Valgus Night Splint

---

### Instructions for care

- The product should be washed before its first use and then on a regular basis.
- The splint can be cleaned using a damp cloth.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

---

### Care

-  Hand wash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

---

### Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

---

## Indications

- Hallux Valgus (bunion)
- Formation of bony lumps
- Deformed big toe
- For stabilisation after bunion surgery

---

## Material composition

- 38% Acrylonitrile butadiene styrene
- 22% Polyurethane (foam)
- 20% Polyamide microfibre
- 18% Polyamide
- 2% Polyurethane

---

## Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

---

## Contraindications

- Consult your doctor before using this product in the following cases:
- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
  - Feeling of numbness and circulatory problems in the area where the product is used
  - Problems with lymphatic drainage and unexplained swelling

---

## Disposal

Please dispose of the product according to local regulations after the end of use.

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---

## Garantie

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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### **Ofa Bamberg GmbH**