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## Instructions for use

---

### Spreizfussbandage

#### Splay Foot Support

---

dynamics®

ofa bamberg

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**Orthopädie**

Orthopaedics

the medical  
compression  
specialist  
since  
**1928**  
made  
in Germany

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---

**Ofa Bamberg.**  
**More than 90 years'**  
**experience with your**  
**health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy. In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive support and travel stockings.

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#### ← Fitting instructions

---

**The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.**

Pull the support over your foot in such a way that the pad is positioned underneath the sole of the foot with the round side pointing towards the toes. The pad is correctly positioned if it is directly behind the metatarsal bones.

#### Intended Use

---

**Dear user,**

**The Dynamics Splay Foot Support** applies targeted pressure to the midfoot region, relieving strain on the area and reducing pain. In addition, an integrated pad provides support to the transverse arch of the foot.

**Ofa Bamberg wishes you a speedy recovery!**

### **Important Information**

---

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product is fitted correctly. Check the product to ensure it is working and check it for wear and damage (such as breaks, cracks or distortion) before each use. Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

---

## Important information

### Splay Foot Support

---

#### Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Remove the pad from its sleeve before washing. It can be cleaned with a damp cloth. To reinsert the pad correctly after washing, the curved side must point towards the sole of the foot.
- Wash the product by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well. Squeeze the water out carefully without wringing the product.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

---

#### Care

-  Hand wash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

## Reporting obligation

---

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

## Indications

---

Splay foot and painful consequences of this in the mid-foot

## Material Composition

---

- 45 % Nylon
- 30 % Rubber
- 20 % Elastane
- 3 % Polypropylene
- 2 % Polyester

This product contains Latex, which can cause allergic reactions.

## Side effects

---

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

---

- Consult your doctor before using this product in the following cases:
- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
  - Feeling of numbness and circulatory problems in the area where the product is used
  - Problems with lymphatic drainage and unexplained swelling

## Disposal

---

Please dispose of the product according to local regulations after the end of use.

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---

## **Guarantee**

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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### **Ofa Bamberg GmbH**