Knöchelbandage

Ankle Support

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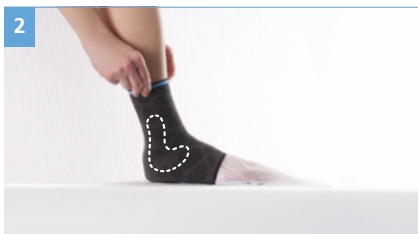
dynamics®plus
Lumbalbandage
Lumbar Support

dynamics®plus
Handgelenkbandage
Wrist Support

dynamics®plus
Epicondylitisbandage
Epicondylitis Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex® comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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dynamics[®] plus
Ellenbogenbandage
 Elbow Support

dynamics[®] plus
Kniebandage
 Knee Support

dynamics[®] plus
Achillesbandage
 Achilles Tendon Support

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

The ankle support can be put on quickly and easily using the enclosed fitting aid. This is done by slipping your foot into the opening of the fitting aid.

2

Now pull the support over the foot with the longitudinal seam facing the rear. The support is sitting correctly if the silicone pad is surrounding the ankle from below.

3

Finally, remove the fitting aid by pulling it out of the support over the toes towards the front.

The Dynamics Plus Ankle Support

stabilises the ankle joint by means of its compressive effect. Massaging silicone pads help reduce swelling rapidly. For a pleasant wearing feeling the extremely thin, highly elastic Arthroflex® comfort zone is incorporated in the sensitive area of the talus.

Please note

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists.
→ see page 2.
- The product should be used only for the applications shown in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Wear the product directly on your skin. It may only be worn on intact skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

Care instructions

- The product should be washed before being used for the first time.
 - Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
 - Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
 - In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area.
- These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging. Please dispose of the product according to local specifications after the end of use.

Obligation of notification

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- tendinitis
- aftercare following damage to the ankle and the ligaments, e. g. contusions, sprains, capsule lacerations, collateral ligament ruptures (fibular ligament rupture)
- joint wear (arthrosis), joint inflammation (arthritis)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

Material composition

71 % polyamide
29 % elastane

Care

- Careful wash programme
 - Do not bleach
 - Do not put into a dryer
 - Do not iron
 - Do not dry clean
- Do not use softener!

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Warranty

When used as intended and looked after properly, the useful life of the product is six months. The product must be treated carefully in accordance with these instructions. Incorrect use or inappropriate modifications to the product may alter the performance, safety and functionality of the product and may lead to the exclusion of claims against it. The product is design to be used to treat a single patient. It is not designed for repeated use in two or more patients.

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dynamics® plus Ankle Support
We get you moving again quickly.

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Ofa Bamberg GmbH