Air Walker

ofa bamberg

Ofa Bamberg GmbH
Laubanger 20
D-96052 Bamberg
Tel. +49 951 6047-0
Fax +49 951 6047-185
info@ofa.de
www.ofa.de

ofa austria

Ofa Austria
Wasserfeldstr. 15
A-5020 Salzburg
Tel. +43 662 848707
Fax +43 662 849514
info@ofaaustria.at
www.ofaaustria.at

ofa bamberg

dynamics®

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EN english	Air Walker	Adjustment by experts	6
		Important notes	10
		Warranty	20

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Ofa Bamberg. **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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Fitting the optional wedge set

(by healthcare professionals only)

Certain conditions require the foot to be stabilised with the toes pointed (plantar flexion). For this purpose a set of wedges can be requested that is inserted into the cuff of the high version of the Walker. The set of wedges will be fitted to your requirements by trained experts. The set consists of five wedges that are connected by means of adhesive points. For size S wedges 1 to 3 must be used, for size M wedges 2 to 4, and for sizes L and XL wedges 3 to 5. Three wedges on top of each other correspond to a pointed toe position of approx. 28°.

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The Dynamics Air Walker provides optimum immobilisation for the foot and the lower leg after injuries or operations. Individually regulated air chambers are

incorporated for the correct embedding of the ankle area. The non-slip tread and the rounded outsole make walking safe and pleasant.

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

Please note: The hygienic cover provided is for indoor wear only and is to protect upholstered furniture and beds.

1

Please put on the Walker while sitting, and, if necessary, put on the stocking provided. To put on the Walker, open all Velcro fasteners and the textile cuff. Place your leg in the cuff so that your heel is in contact with the back of the Walker.

2

First close the Velcro fastener of the cuff over your instep, and then close the toe flap. Then close the cuff at your shin. Now put on the front plastic splint by Velcro fastening it to the cuff.

3

Then close the Velcro tapes from top to bottom by passing them through the corresponding eyelet and then Velcro fastening them. Please ensure that to prevent pinching the product is secure but not too tight. The Walker must feel comfortable.

4

Finally, the air cushions are adjusted to the required pressure. In order to do this, attach the light tip ("In") of the provided air pump to the blue valve and inflate the air cushions as required. Please do not over-expand the cushions when doing this. Before taking off the Walker the air should be let out again by attaching the black pump tip ("Out") to the valve and pumping out the air. This will make the Walker easier to put on and take off, and will avoid over-expanding the air cushions.

english

Care instructions

- The product should be washed before being used for the first time.
 - Detach the removable padding from the splints before washing. Please be sure to avoid pulling off the Velcro strips on the splints. The splints can be cleaned with a moist cloth.
 - Wash the textiles by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
 - Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
 - In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.
 - Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.
-
-  Hand wash
 -  Do not bleach
 -  Do not put into a dryer
 -  Do not iron
 -  Do not dry clean
- Do not use softener!**

Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. → see page 6
- If the product has to be worn in combination with other products, then please discuss this in advance with your doctor.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then contact your doctor or specialist supplier immediately.
- We use a comprehensive quality management system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

Indications

- severe ankle sprains
- stable foot or ankle fractures
- immobilisation before and after operations
- ruptured Achilles tendon (high version of Walker only)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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