Hallux Valgus Schuh Hallux Valgus Shoe

ofa bamberg

Ofa Bamberg GmbH
Laubanger 20
D-96052 Bamberg
Tel. +49 951 6047-0
Fax +49 951 6047-185
info@ofa.de
www.ofa.de

ofa austria

Ofa Austria
Wasserfeldstr. 15
A-5020 Salzburg
Tel. +43 662 848707
Fax +43 662 849514
info@ofaaustria.at
www.ofaaustria.at

ofa bamberg

dynamics®

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Orthopädie
Orthopaedics

the medical
compression
specialist
since
1928
made
in Germany

Ofa Bamberg.
More than 90 years' experience
for your health.

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

To put on the shoe, open all Velcro fasteners. Place your foot into the shoe and close the Velcro fasteners over the instep.

2

Then close the Velcro fastener of the heel strap. Please ensure that the product is secure but not too tight in order to prevent pinching.

The Dynamics hallux valgus shoe assists with your recovery immediately after a Hallux Valgus operation until the foot has healed completely. Its special rounded and stiffened sole makes it easier to roll over the foot. The cushion of the shoe also has soft padding in the front area.

Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. It might be necessary to make adjustments for uneven leg lengths.
- Do not wear the product directly on injured skin.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then contact your doctor or specialist supplier immediately.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

english

Care instructions

- The shoe can be cleaned using a damp cloth. For heavier soiling, use a mild soapy solution.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches or petrol. These substances can affect the material of the product.

Material composition

Top:	synthetic
Sole:	synthetic
Cushion:	leather
	synthetic textile

Care

- ☑ Hand wash
- ✘ Do not bleach
- ✘ Do not dry clean
- ✘ Do not put into a dryer
- ✘ Do not iron

Indications

- after hallux valgus operations
- in the event of problems in the forefoot area

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using this product:

- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- feeling of numbness and circulatory problems in the area of application
- problems with lymphatic drainage as well as unexplained swelling

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.