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Splay foot support

ofa bamberg

dynamics®

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**Orthopädie**

Orthopaedics

the medical  
compression  
specialist  
since  
**1928**  
made  
in Germany

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---

**Ofa Bamberg.  
More than 90 years'  
experience for your  
health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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← **Fitting instructions**

---

**The fitting instructions in pictures can be found in the front cover (2) of the brochure.**

Pull the support over the foot in such a way that the pad is positioned beneath the sole of the foot with the round side pointing towards the toes. The pad is correctly positioned if it is directly behind the metatarsal heads.

---

**Dear user,**

Dynamics foot supports relieve the bones and ligaments of the metatarsus by means of their compressive effect. The special pads of the splayfoot and hallux valgus supports also support the foot's lateral arch. The ideal solution, particularly if the feet are exposed to excessive stress on a daily basis.

**Ofa Bamberg wishes you a speedy recovery!**

## Please note

---

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists.  
→ see page 2.
- The product should be used only for the applications shown in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Wear the product directly on your skin. It may only be worn on intact skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

## Important notes

### Foot supports

#### Care instructions

---

- The product should be washed before first use and regularly thereafter.
- Remove the pad from its sleeve before washing (if present). It can be cleaned with a moist cloth. To reinsert the pad correctly after washing, the pad's curvature must be pointing to the sole of the foot.
- Wash the product by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.
- Please dispose of the product according to local specifications after the end of use.

#### Obligation of notification

---

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

## Indications

---

Splayfoot and painful after-effects in the matatarsus

## Material composition

---

45% nylon

30% rubber 

20% spandex

3% polypropylene

2% polyester

## Care

---

-  Hand wash
-  Do not bleach
-  Do not put into a dryer
-  Do not iron
-  Do not dry clean

Do not use softener!

## Side effects

---

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

---

In the following cases you should consult your doctor before using this product:

- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- feeling of numbness and circulatory problems in the area of application
- problems with lymphatic drainage as well as unexplained swelling

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---

## Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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### **Ofa Bamberg GmbH**