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## Instructions for use

---

### Knielagerungsschiene 0°/20°

### Knee Immobilisation Splint 0°/20°

---

dynamics®

ofa bamberg

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### **Ofa Bamberg.** **More than 90 years' experience** **with your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was founded in 1928, we have set extremely high standards for our products: maximum comfort and optimal efficacy.

In addition to medical supports and orthoses, Ofa Bamberg also offers an extensive range of medical compression stockings and preventive travel and support stockings.

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---

**Intended Use**

**The Dynamics Immobilisation Splint** immobilises your knee joint in an extended (0° version) or flexed (20° version) position. Aluminium stabiliser bars integrated on the sides and at the rear limit the range of

movement of your knee joint, helping to support its healing process, for example after an operation.

**Fitting instructions**

**The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.**

---

**1**

To put the splint on, open all of the Velcro fasteners and open the cuff up completely. The side parts are individually adjustable. Adjust them so that the cuff completely encloses the leg.

---

**2**

When putting the cuff on, please be aware that it gets narrower at the lower end. Put the splint against the back of your leg. First close the Velcro fastener directly below the knee by pulling the strap through the corresponding eyelet and fastening it tightly. Then close the Velcro fastener above the knee. The splint is correctly positioned if the kneecap is in the corresponding opening.

---

**3**

Close all of the Velcro fasteners one after the other. If necessary you can then adjust the Velcro fasteners until the splint is correctly positioned. Ensure that the product is secure but not too tight so as to prevent pinching.

---

 **Important Information**

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Check the product to ensure it is working and check it for wear and damage before each use. You should also ensure it is fitted correctly.  
Do not use a product that is damaged.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

# english

## Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Remove the splints before washing.
- Close all of the Velcro fasteners and wash the product separately by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well. Squeeze the water out of the product gently but do not wring it out.
- Mould the product into the right shape and let it dry in the air. Do not dry the product in a tumble dryer, on a radiator or in the direct sunlight, and never iron it.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

### Care

-  Hand wash
  -  Do not bleach
  -  Do not tumble dry
  -  Do not iron
  -  Do not dry clean
- Do not use fabric softener!**

## Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

## Indications

- Immobilisation of the knee after injuries and operations

## Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

- Consult your doctor before using this product in the following cases:
- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
  - Feeling of numbness and circulatory problems in the area where the product is used
  - Problems with lymphatic drainage and unexplained swelling

## Material Composition

Cuff:  
 43 % Polyamide  
 36 % Polyester  
 14 % Polyurethane  
 7 % Cotton  
 Stabilising splints:  
 Aluminium

## Disposal

Please dispose of the product according to local regulations after the end of use.

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---

### **Garantie**

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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### **Ofa Bamberg GmbH**