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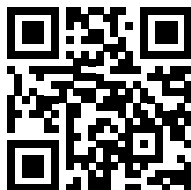
Instructions for use

ofa[®]

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english

Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

1 To put the shoe on, open all of the Velcro fasteners. Put your foot in the shoe and close the Velcro fasteners over the instep.

2 Then close the Velcro fastener on the heel strap. Ensure that the product is secure but not too tight so as to prevent pinching.

Intended Use

The Dynamics Hallux Valgus Shoe (Comfort) stabilises and partially immobilises the foot. The stiff outsole limits the range of movement of the midfoot and forefoot, facilitating healing processes, for example after an operation.

Important information

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss

this with your doctor first.

- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- The shoe can be cleaned with a damp cloth. If it is very dirty you can use mild soapy water for this.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches or petrol on it. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

- ☞ Hand wash
- ☒ Do not bleach
- ☒ Do not dry clean
- ☒ Do not tumble dry
- ☒ Do not iron

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

- After hallux valgus operations
- In the case of diseases in the forefoot area

Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves. There can be local temperature increases. If this happens, briefly elevate the foot and open up the product to avoid heat accumulating.

Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

Material composition

Hallux valgus shoe:

46 % PUR
31 % TPU
6 % PU
4 % PES
4 % SBK
4 % PA
3 % PE
2 % Cotton
PET
CV

Hallux valgus shoe, comfort:

49 % PUR
30 % TPU
6 % PA
4 % SBK
4 % PE
3 % PU
2 % PES
1 % Cotton
1 % PET
CV

Disposal

Please dispose of the product according to local regulations after the end of use.

Guarantee

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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Ofa Bamberg GmbH