Ellenbogenbandage
Elbow Support

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dynamics® plus
Lumbalbandage
Lumbar Support

dynamics® plus
Handgelenkbandage
Wrist Support

dynamics® plus
Epicondylitisbandage
Epicondylitis Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex® comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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dynamics[®]plus
Kniebandage
Knee Support

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

Pull the support over the arm in such a way that the silicone pads are sitting on the underarm, and the thin, soft Arthroflex® comfort zone is covering the crook of the arm.

2

In order to reinforce the effectiveness of the support, the separate Velcro strap can be attached at the underarm. This is done by attaching the narrow Velcro fastener of the strap to the frieze band on the support.

3

Now lead the strap around the arm and close it using the wide Velcro fastener. Please ensure that the product is secure but not too tight in order to avoid pinching.

The Dynamics Plus Elbow Support

relieves the elbow joint as a result of its compressive effect. Integrated silicone pads relieve the elbow entheses by exerting gentle pressure. A detachable Velcro strap reinforces the effectiveness of the support during the acute phase of an ailment.

Please note

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists.
→ see page 2.
- The product should be used only for the applications shown in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Wear the product directly on your skin. It may only be worn on intact skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

Care instructions

- The product should be washed before being used for the first time.
- Close all Velcro fasteners and wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging. Please dispose of the product according to local specifications after the end of use.

Obligation of notification

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- elbow irritation (chronic, after injuries or operations)
- joint degeneration (arthrosis)
- tennis elbow and golfer's elbow

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

Material composition

65 % polyamide
 26 % elastane
 5 % cotton
 2 % polyester
 2 % polyurethane

Care

- Delicate wash programme
 - Do not bleach
 - Do not put into a dryer
 - Do not iron
 - Do not dry clean
- Do not use softener!**

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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dynamics® plus Elbow Support
We get you moving again quickly.

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Ofa Bamberg GmbH