PUSH ORTHO ANKLE BRACE AEQUI JUNIOR

Please read this product information carefully before using the Push Aequi Junior sorgfältig durch.

INTRODUCTION

Push products provide effective solutions for a variety of disorders of the locomotor apparatus. The products can also be used as a preventive measure after previous injuries of the joints.

The Push ortho ankle brace Aequi Junior is a Push Braces product from the so-called ortho category. Push ortho products are innovative and are carefully manufactured. They aim to provide the highest level of comfort, maximum support and effective protection. The Aequi Junior ankle brace was especially designed for children with unstable ankles.

The Aequi Junior is characterised by a high degree of support, effectiveness, comfort and quality. The product provides enhanced stability and helps control the pain. A major advantage of wearing this brace is that it increases load bearing capacity. This restores the ankle joint's full functionality more quickly, and in particular more safely.

PRODUCT CHARACTERISTICS

The Aequi Junior is an effective ankle brace for children (or adults with a small instep) aged between 6 and 12, with an instep of 23-27 cm (shoe sizes 30-37). This ankle brace is suitable for both the left and right ankle and is available in a single size.

The brace offers a great balance between effective mechanical ankle support and optimal wearer comfort. It consists of a sturdy foot shell and a set of elastic and non-elastic straps. The Aequi Junior provides the required support of the ligaments. Elastic straps also provide pressure around the ankle joint.

The inversion-eversion movement (inward-outward movement of the foot) is restricted, while plantar/dorsal flexion (bending and stretching) is allowed, for adequate freedom. This means that no inhibition is felt during normal walking and running.

The numbers on the straps allow for an easy fitting of the brace.

Tests have shown that children are capable of fitting the brace themselves. For a proper functioning of the brace, however, it is important that parents or carers initially observe and provide any help needed and inspect whether the Aequi Junior has been fitted correctly.

Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. When children play indoors, the Aequi Junior can also be worn without a shoe, while retaining the function of stabilisation.

INDICATIONS

- In the case of early functional therapy for injuries to the medial or lateral ankle ligaments
- Follow-up treatment of conservative or operated ankle fractures
- In the case of residual or other instability (increased tendency to sprain)
- To prevent (recurrence of) ankle ligament injuries

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Always applying the brace too lightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

Always follow the advice of your doctor or therapist.

FITTING

The hook and loop fastenings are marked with symbols. These are to help you fasten the hook and loop material in the correct sequence. First ☐ on ☐, then ☐ on ☐, etc.

- The Push Aequi Junior is worn over the sock.
- Place the brace around the foot. Lay the long straps to the sides.
- Fix the diagonal straps ① and ② and the shell closely around the ankle (Figure A).
- Wrap the long elastic strap ③ diagonally across the instep and fasten it on the hook and loop material ④.
- Wrap the long elastic strap ⑤ diagonally across the instep and fasten it on the hook and loop material ⑥.

If the brace is worn by a child, always check whether it has been fitted properly. If the product should be left on during the night, it should be fitted less tightly.

WASHING INSTRUCTIONS

Before washing, close the hook and loop fasteners to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C on a delicate fabrics cycle. Do not use any bleaching agents.

Hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Aequi Junior requires a correct diagnosis of the causes of the complaint. Always consult your GP, a physiotherapist or a specialist before using the brace.
- Push Braces should be designed to be worn on an undamaged skin. Damaged skin must first be covered with suitable material.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before purchasing, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Check your Push brace each time before use for signs of wear or ageing of components or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.
- To promote the useful life of the brace, it should not be exposed to temperatures above 70 °C.
- Within the framework of EU regulation 2017/745 on medical devices, the user must report any serious incident involving the product to the manufacturer and to the competent authority of the Member State in which the user is established.
- Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. Be careful not to slip when the brace is worn without a shoe.

ANY QUESTIONS?

For any questions concerning the product, please consult the reseller who provided the brace, or send an e-mail to info@push.eu.

Push® is a registered trademark of Nea International bv

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