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Instructions for use

ofa[®]

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english

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

- 1** Pull the support over the arm in such a way that the silicone pads are sitting on the underarm, and the thin, soft Arthroflex® comfort zone is covering the crook of the arm.
- 2** In order to reinforce the effectiveness of the support, the separate Velcro strap can be attached at the underarm. This is done by attaching the narrow Velcro fastener of the strap to the frieze band on the support.
- 3** Now lead the strap around the arm and close it using the wide Velcro fastener. Please ensure that the product is secure but not too tight in order to avoid pinching.

Purpose

The product is used to compress the elbow joint; its compressive effect and the incorporated pads stabilise the elbow joint.

⚠ Important notes

- The initial fitting and positioning of the product as well as training in its proper, safe use should be carried out by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Close all Velcro fasteners and wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning prod-

ucts, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.

- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

- 🧺 Delicate wash cycle
- ✖ Do not bleach
- 🧺 Do not tumble dry
- ✖ Do not iron
- ✖ Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Intended users and target patient group

The target patient group includes all persons with one of the indications mentioned above who have no contraindications. Selection of the appropriate size, as well as the initial fitting and application, must be carried out by trained medical or orthopaedic professionals. After instruction by this specialist personnel, the product can also be used safely by laypersons.

Indications

- Elbow irritation (chronic, after injuries or operations)
- Joint degeneration (arthrosis)
- Tennis elbow and golfer's elbow

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

In the following cases you should consult your doctor before using this product:

- Skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- Feeling of numbness and circulatory problems in the area of application
- Problems with lymphatic drainage as well as unexplained swelling

Material composition

51 % Polyamide
33 % Silicone
7 % Elastane
3 % Viscose
3 % Polyester
2 % Cotton
1 % PU
Aluminium

Disposal

Please dispose of the product according to local regulations after the end of use.

Guarantee

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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Ofa Bamberg GmbH