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Instructions for use

ofa[®]

Produktinformationen und Gebrauchsanweisung - online
Product information and instructions for use - online

dynamics[®]

ROM Knieschiene

REF 44 6027

EN ROM Knee Splint

NL ROM-kniepalk

IT Stecca per Ginocchio ROM

deutsch	4
english	5
nederlands	7
italiano	9

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english

Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

- 1** To put the product on, open all of the Velcro fasteners and the fabric cuff. When putting the product on, please be aware that the cuff gets narrower at the lower end. Place the splint against the back of

the leg so that the ROM joints are positioned in line with the middle of the knee.

2 First close the wide cuff on the lower leg, then the one on the thigh. Then close the two narrow cuffs.

3 Then close the Velcro straps in the same order by inserting them through the eyelet provided and fastening them tightly. If necessary you can then adjust the cuff and the Velcro fasteners until the splint is correctly positioned. Ensure that the product is secure but not too tight so as to prevent pinching.

Fitting of the ROM Knee Splint (only by experts) see figures on page 2

A The ROM Knee Splint is individually tailored to you by trained experts. The length of the lateral splints can be adjusted to any one of five levels at the thigh and lower leg. This is done by pressing and holding the oval, grey button and the splint is shortened or extended as required.

B The transparent cover can be rotated and opened to adjust the ROM joints by pushing the cap outwards and flipping it open. The metal pins are used to adjust the flexion and extension. Both joints should always be set to the same levels. The cover is closed again once the adjustment is complete.

Intended Use

The product is designed to stabilise and completely or partially immobilise the knee joint. Metal stabiliser bars and an adjustable joint hinge limit the degrees of freedom of the knee joint.

Important information

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.

- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Detach the removable padding from the splints for washing. Ensure that you do not pull the Velcro strips off the splints too. The splints can be cleaned with a damp cloth.
- Close all Velcro fasteners and wash the fabric by hand. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Hand wash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

– After injuries or operations to immobilise the knee joint with subsequent mobilisation in defined ranges of motion

Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
 - Feeling of numbness and circulatory problems in the area where the product is used
 - Problems with lymphatic drainage and unexplained swelling
-

Material composition

45 % Nylon
21 % Aluminium
15 % Steel
8 % PU foam
6 % Polyester
3 % Cotton
2 % POM
Silicone

Disposal

Please dispose of the product according to local regulations after the end of use.

Guarantee

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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Ofa Bamberg GmbH