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### **Ofa Bamberg.** **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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---

**Dear user,**

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

---

**The Dynamics knee immobilisation splint**

provides optimum immobilisation of the knee joint in the straight (0° version) or bent position (20° version). The skin-friendly terry cloth material on the inside guarantees enhanced wearing comfort. The open design facilitates putting on and taking off the splint, thus simplifying personal hygiene and wound treatment.

---

**Fitting instructions**

**The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.**

---

1

In order to put on the splint, open all Velcro fasteners and open up the cuff completely. The lateral parts are individually adjustable. Adjust them so that the cuff encloses the leg completely.

---

2

When putting the cuff on, please note that it becomes narrower at the lower end. Put the splint against the back of your leg. First close the Velcro fastener directly below the knee by pulling the tape through the corresponding eyelet and fastening it down. Then close the Velcro fastener above the knee. The splint is positioned correctly when the kneecap is in the corresponding opening of the cuff.

---

3

Now close all Velcro fasteners one after the other. The Velcro fasteners can then, if necessary, be adjusted until the splint is correctly positioned. Please ensure that the product is secure but not too tight in order to prevent pinching.

# english

## Care instructions

- The product should be washed before being used for the first time.
- Wash the product by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.
- Store the product in a cool dry

place away from sunlight and heat and preferably in the original packaging.

- 
-  Hand wash
  -  Do not bleach
  -  Do not put into a dryer
  -  Do not iron
  -  Do not dry clean
- Do not use softener!**

## Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. The aluminium splint can be moulded when cold.
- If the product has to be worn in combination with other products, then please discuss this in advance with your doctor.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then contact your doctor or specialist supplier immediately.
- We use a comprehensive quality management system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

## Indications

- immobilisation of the knee joint after injuries and operations.

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

In the following cases you should consult your doctor before using this product:

- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
- feeling of numbness and circulatory problems in the area of application
- problems with lymphatic drainage as well as unexplained swelling

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---

### Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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