Lumbalbandage Lumbar Support

ofa bamberg

Ofa Bamberg GmbH
Laubanger 20
D-96052 Bamberg
Tel. + 49 951 6047-0
Fax + 49 951 6047-185
info@ofa.de
www.ofa.de

ofa austria

Ofa Austria
Wasserfeldstr. 15
A-5020 Salzburg
Tel. + 43 662 848707
Fax + 43 662 849514
info@ofaaustria.at
www.ofaaustria.at

dynamics®

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dynamics®
We'll get you moving
in no time.

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Ofa Bamberg. More than 85 years' experience for your health.

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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Dear user, The Dynamics lumbar support assists your lumbar spine by means of its compressive effect. The incorporated air padding can be individually adjusted in an optimum way via the integrated pump.

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

Before putting on the support, please open all Velcro fasteners. Place the support around the body and close the Velcro fastener of the inner strap in front of your stomach so that it is comfortably secure. The lower edge (marked by a small loop) should be in the lowest area of the lumbar spine. Please ensure that the loop is centrally positioned.

2

Now close the outer tension straps by pulling them towards the front and Velcro fastening the left and then the right strap to the body belt. The pressure can be individually adjusted.

3

Next inflate the air padding by pressing the blue air pump several times until the padding sits comfortably and provides the required support. While inflating the padding, the valve must be closed (i. e. turned towards "Inflate"). In order to release air from the padding, turn the valve towards "Deflate" and then close it again. Do not overinflate the padding with air and do not close the valve too firmly.

The gel cushion provided can be used in the support for heat therapy in the case of back pain.

Ofa Bamberg wishes you a speedy recovery!

Use with gel cushion

Please consult your doctor before using the gel cushion provided. For heat therapy, place the cushion in the microwave for 20 seconds at a low setting. Then check the temperature of the cushion with your forearm. If necessary, repeat the procedure until the required temperature has been reached. Velcro fasten the heated gel cushion to the air padding using Velcro (figure 4) and fit the support as described.

Please note

-
- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff.
 - Wear the product over a thin close-fitting shirt. If it has to be worn in combination with other products, please discuss this with your doctor beforehand.
 - During long periods of rest (e. g. when sleeping) the product should be removed.
 - If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
 - We use a comprehensive quality assurance system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

Care instructions

- The product should be washed before being used for the first time.
- Never immerse the air pump and the valve in water when washing.
- Wash the product by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- The gel cushion can be cleaned with a damp cloth.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Indications

- acute and chronic pain in the lumbar and sacral areas of the spine.
- lumbago
- degenerative changes to the lumbar spine

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

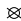
Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

Material composition

Support:
 40 % polyamide
 26 % polyester
 26 % elastane
 3 % cotton
 5 % other
 Gel cushion:
 55 % glycerine
 37 % acrylamide
 4 % vinyl
 4 % other

Care

-  Hand wash
-  Do not bleach
-  Do not put into a dryer
-  Do not iron
-  Do not dry clean

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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