---

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Instructions for use

**ofa**<sup>®</sup>

---

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## english

### Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

- 1** Before putting on the support, open all of the Velcro fasteners and check the position of the pad. The tip of the pad should point downwards and

should be able to be fastened to the desired height on the inside of the support using Velcro.

- 2** The support comes with hand straps to make it easier to put on. First insert your left hand through the inner strap and wrap the support around your body. Then insert your right hand through the outer strap and fasten the Velcro in front of your stomach.

- 3** Now close the additional tension straps by pulling them forwards and fastening them to the body part. The pressure can be individually adjusted.

## Intended Use

The product is used to compress the abdominal area and the lumbar region. The compressive effect and the reinforcing elements stabilise the lumbar region. Additional reins enable targeted and individual pressure regulation in the abdominal cavity. The pad supports local blood circulation.

## ⚠ Important information

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- Wear the product over a thin, close-fitting shirt. If it is to be worn in combination with other products, please discuss this with your doctor first.
- Unless otherwise prescribed, the product should be removed during long periods of rest (e.g. when sleeping).
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

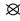
---

### Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Remove the pad from its sleeve before washing. It can be cleaned with a damp cloth. To insert the pad again correctly, the side with the nubs must face backwards when it is inserted into the sleeve (Figure 4).
- Close all of the Velcro fasteners and wash the product separately in a wash bag at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well. The product can be put on a gentle spin cycle, but please do not wring it out.
- Mould the product into the right shape and let it dry in the air. Do not dry the product in a tumble dryer, on a radiator or in the direct sunlight, and never iron it.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

---

### Care

-  Delicate wash cycle
  -  Do not bleach
  -  Do not tumble dry
  -  Do not iron
  -  Do not dry clean
- Do not use fabric softener!

---

### Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

---

### Indications

- Acute and chronic pain in the lumbar and sacral regions of the spine
- Degenerative changes to the lumbar spine  
Only the 28 cm version:
- Minor spondylolisthesis
- Aftercare of fractures of the lumbar spine
- Spondylolysis

---

### Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

---

### Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

---

### Material composition

- 54 % Polyamide
- 18 % Elastodiene
- 10 % TPU
- 8 % Steel
- 6 % Polyester
- 2 % Polypropylene
- 1 % Elastane
- 1 % PVC

**This product contains Latex, which can cause allergic reactions.**

---

### Disposal

Please dispose of the product according to local regulations after the end of use.

---

## Guarantee

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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**Ofa Bamberg GmbH**