
ROM Knieschiene ROM Knee Splint

ofa bamberg

Ofa Bamberg GmbH
Laubanger 20
D-96052 Bamberg
Tel. + 49 951 6047-0
Fax + 49 951 6047-185
info@ofa.de
www.ofa.de

ofa austria

Ofa Austria
Wasserfeldstr. 15
A-5020 Salzburg
Tel. + 43 662 848707
Fax + 43 662 849514
info@ofaaustria.at
www.ofaaustria.at

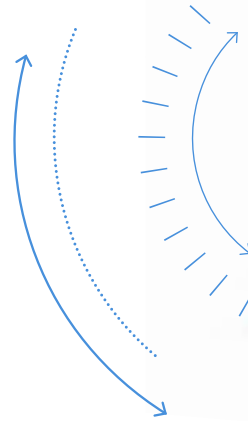ofa bamberg

dynamics®

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dynamics®
We'll get you moving
in no time.

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Ofa Bamberg. **More than 90 years' experience** **for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive support and travel stockings.

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Adjusting the ROM knee splint

(by healthcare professionals only)

The ROM knee splint is individually adjusted for you by trained staff. The length of the lateral splints can be adjusted in five stages at the thigh and at the lower leg. This is done by pressing and holding down the oval, grey button and lengthening or shortening the splint as required.

→ see figure A, page 6

The transparent cover can be rotated and opened to adjust the ROM joints by pushing the closing cap outwards and flipping it open. The flexion and extension are adjusted using the metal pins. Both joints must be set to the same settings. Close the cover again after adjusting.

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

Ofa Bamberg wishes you a speedy recovery!

The Dynamics ROM knee splint stabilises the knee in an individually definable position after injuries or operations. For an early mobilisation of the knee joint a suitable range of motion (ROM) can be set via the ROM joints. Soft textile cuffs provide a high degree of wearing comfort, and the open construction is easy to put on and take off.

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

To put on the product, open all Velcro fasteners and the textile cuffs. When putting on the splint, please note that the cuffs become narrower at the lower end. Place the splint against the back of the leg so that the ROM joints are positioned in line with the centre of the knee.

2

Now close the wide cuff at the lower leg, then the one at the thigh. Then close the two narrow cuffs.

3

At this point fasten the Velcro tapes in the same order, pulling them through the corresponding eyelets and then fastening them. The cuffs and the Velcro fasteners can, if necessary, be readjusted afterwards until the splint sits perfectly. Please ensure that the product is secure but not too tight in order to prevent pinching.

Please note

- The initial adjustment and strap-
ping on of the product must be
carried out by qualified specialist
staff. → [see page 6](#)
- If the product has to be worn in
combination with other products,
then please discuss this in ad-
vance with your doctor.
- If you experience an unpleasant
sensation, intense pain or any
other problem when wearing the
product, then contact your doctor
or specialist supplier immediately.
- We use a comprehensive quality
management system to check our
products. However, should there
still be reasons for complaint,
please contact your specialist sup-
plier.

english

Care instructions

- The product should be washed before being used for the first time.
- Detach the removable padding and Velcro tapes from the splints before washing. Please be sure to avoid pulling off the Velcro strips on the splints. The splint can be cleaned with a moist cloth.
- Wash the textiles by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can irritate the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Indications

- for immobilising the knee joint after injuries or operations with subsequent mobilisation within a defined range of motion

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

Material composition

45 % polyamide
35 % polyurethane
20 % cotton

Care

-  Hand wash
 -  Do not bleach
 -  Do not put into a dryer
 -  Do not iron
 -  Do not dry clean
- Do not use softener!

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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