Epicondylitisbandage

Epicondylitis Support

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dynamics[®] plus
Lumbalbandage
Lumbar Support

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The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex[®] comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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dynamics[®]plus
Kniebandage
Knee Support

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics Plus product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found in the front cover (2) of the brochure.

1

To put on the support, open the elastic strap. Pull the wider end of the support over the forearm, not over the elbow. With tennis elbow the pad must lie on the dorsal side of the forearm, and with golfer's elbow it must lie on the ventral side. Please wear the support directly on the skin – this is the only way to obtain the desired effect.

2

The separate Velcro strap can be worn in order to obtain a better fit. Attach the narrow hook strip of the strap to the loop strip on the support. Now lead the strap around the arm and fasten it loosely. Please ensure that the product is secure but not too tight in order to avoid pinching.

The Dynamics Plus Epicondylitis

Support targets the causes of tennis elbow and golfer's elbow. The large Plus Pad absorbs the vibration that is caused by hand and finger movements. This prevents irritation from being transferred to the elbow, so that the inflammation of the muscle attachments can abate.

Please note

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists.
→ see page 2.
- The product should be used only for the applications shown in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Wear the product directly on your skin. It may only be worn on intact skin. If it has to be worn in combination with other products, please discuss this with your doctor beforehand.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

Care instructions

- The product should be washed before being used for the first time.
- Before washing, remove the pad from the support and close the Velcro fastener of the strap. The pad must not be machine washed. Please clean the pad with a damp cloth or handwash with a mild soapy solution.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area. These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging. Please dispose of the product according to local specifications after the end of use.

Obligation of notification

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

Indications

- tennis elbow (Epicondylitis humeri radialis)
- golfer's elbow (Epicondylitis humeri ulnaris)

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
 - feeling of numbness and circulatory problems in the area of application
 - problems with lymphatic drainage as well as unexplained swelling

Material composition

Support:
78 % polyamide
22 % elastane
Pad:
100 % polyurethane
Laminate:
polyester

Care

- Careful wash programme
 - Do not bleach
 - Do not put into a dryer
 - Do not iron
 - Do not dry clean
- Do not use softener!**

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Warranty

When used as intended and looked after properly, the useful life of the product is six months. The product must be treated carefully in accordance with these instructions. Incorrect use or inappropriate modifications to the product may alter the performance, safety and functionality of the product and may lead to the exclusion of claims against it. The product is design to be used to treat a single patient. It is not designed for repeated use in two or more patients.

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dynamics® plus Epicondylitis Support
We get you moving again quickly.

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Ofa Bamberg GmbH