# Hallux Valgus Schuh

Hallux Valgus Shoe

ofa bamberg

dynamics®

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**Orthopädie**  
Orthopaedics

since  
**1928**  
the medical  
compression  
specialist  
made  
in Germany

**Ofa Bamberg.**  
**More than 90 years' experience**  
**for your health.**

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive health and travel stockings.

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---

**Dear user,**

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

**Ofa Bamberg wishes you a speedy recovery!**

**Fitting instructions**

---

**The fitting instructions in pictures can be found in the front cover (2) of the brochure.**

**1**

---

To put on the shoe, open all Velcro fasteners. Place your foot into the shoe and close the Velcro fasteners over the instep.

**2**

---

Then close the Velcro fastener of the heel strap. Please ensure that the product is secure but not too tight in order to prevent pinching.

---

**The Dynamics hallux valgus shoe**

assists with your recovery immediately after a Hallux Valgus operation until the foot has healed completely. Its special rounded and stiffened sole makes it easier to roll over the foot. The cushion of the shoe also has soft padding in the front area.

---

**Please note**

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists. → see page 2.
- The product should be used only for the applications shown in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Do not wear the products directly on injured skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality management system to check our products. However, should there still be reason for complaint, please contact your specialist supplier.

## Care instructions

- The product should be cleaned before first use and regularly thereafter.
- The shoe can be cleaned using a damp cloth. For heavier soiling, use a mild soapy solution.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches or petrol. These substances can affect the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.
- Please dispose of the product according to local specifications after the end of use.

## Obligation of notification

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

## Indications

- after hallux valgus operations
- in the event of problems in the forefoot area

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - feeling of numbness and circulatory problems in the area of application
  - problems with lymphatic drainage as well as unexplained swelling

## Material composition

### Hallux valgus shoe:

46% PUR	4% PA
31% TPU	3% PE
6% PU	2% Cotton
4% PES	PET
4% SBK	CV

### Hallux valgus shoe comfort:

49% PUR	3% PU
30% TPU	2% PES
6% PA	1% Cotton
4% SBK	1% PET
4% PE	CV

## Care

-  Hand wash
-  Do not bleach
-  Do not dry clean
-  Do not put into a dryer
-  Do not iron

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#### **Warranty**

---

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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### **Ofa Bamberg GmbH**