## Achillesbandage

Achilles Tendon Support

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dynamics® plus  
**Lumbalbandage**  
Lumbar Support

dynamics® plus  
**Handgelenkbandage**  
Wrist Support

dynamics® plus  
**Epicondylitisbandage**  
Epicondylitis Support

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### The intelligent orthopaedic support “made in Germany”

Innovative functionality, combining quality and design – this is the idea behind the development of Dynamics Plus Supports. They offer effective support for your joints with optimal medical efficacy. Arthroflex® comfort zones provide additional wearer comfort, protecting the sensitive areas of the joints. The organic honeycomb design enhances their function and subtly reflects the anatomy.

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**Dear user,**

Please take a moment to read through this product information carefully to ensure optimal effectiveness and safe use of your Dynamics Plus product.

**Ofa Bamberg wishes you a speedy recovery!**

**Fitting instructions**

---

**The fitting instructions in pictures can be found in the front cover (2) of the brochure.**

**1**

The support can be put on quickly and easily using the enclosed donning aid. This is done by slipping your foot into the opening of the donning aid.

**2**

Now put the support on your foot, the long seam towards the back. The support is positioned correctly if the silicone pad is centred over your Achilles tendon and the Arthroflex zone is at the front of the ankle.

**3**

Lastly remove the donning aid, pulling it forward out of the support and over your toes.

**4**

To remove, insert both thumbs into the top edge and pull the support off over the heel.

**The compression effect of the Dynamics Plus Achilles Tendon Support**

relieves the area around the Achilles tendon.

The massaging silicone truss pad helps to quickly reduce swelling. The extra thin, extremely stretchy Arthroflex® comfort zone is built into the sensitive area at the front of the ankle for comfort.

**Please note**

---

- The initial fitting and positioning of the product as well as training in its proper, safe use should take place by trained medical/orthopaedic specialists. → see page 2.
- The product should be used only for the applications shown in the adjacent column.
- Before each use, check that the product is working properly and check it for wear and damage, and ensure that it is fitted correctly. If the product is damaged, it should no longer be used.
- Wear the product directly on your skin. It may only be worn on intact skin. If it has to be worn in combination with other products, then please discuss this in advance with your doctor.
- Unless otherwise prescribed, the product should be removed before resting for longer periods (e.g. whilst sleeping).
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then remove it and contact your doctor or specialist supplier.
- Our products are subject to thorough quality control. However, should there still be reasons for complaint, please contact your specialist supplier.

## Care instructions

- The product should be washed before first use and regularly after use.
- Wash the product separately in a small wash bag at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. A gentle spin can be used for this product but please do not wring it out.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the usage area.
- These substances can irritate the material of the product.
- Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging. Please dispose of the product according to local specifications after the end of use.

## Obligation of notification

In accordance with statutory provisions in the EU, patients and users are obliged to report any serious incident while using a medical device without delay to both the manufacturer and the competent national authority (in Germany the BfArM, the Federal Institute for Drugs and Medical Devices).

## Indications

- Chronic or post-traumatic irritation in the Achilles heel area (e. g. achillodynia)
- Irritation with Haglund's deformity
- Post-operative (e. g. aftertreatment after a ruptured Achilles heel)
- Mucous membrane inflammations (e. g. bursitis subachillea)

## Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

## Contraindications

- In the following cases you should consult your doctor before using this product:
- skin conditions or injuries in the area of application, particularly where signs of inflammation are present, such as redness, temperature increase or swelling
  - feeling of numbness and circulatory problems in the area of application
  - problems with lymphatic drainage as well as unexplained swelling

## Material composition

42 % silicone  
 41 % polyamide  
 6 % elastane  
 3 % viscose  
 3 % polyester  
 3 % cotton  
 2 % polyurethane  
 0,1 % aluminium

## Care

-  Careful wash programme
  -  Do not bleach
  -  Do not put into a dryer
  -  Do not iron
  -  Do not dry clean
- Do not use softener!**

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---

### **Warranty**

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.

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dynamics® plus Achilles Tendon Support

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### **Ofa Bamberg GmbH**