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dynamics®
We'll get you moving
in no time.

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Orthopädie
Orthopaedics

the medical
compression
specialist
since
1928
made
in Germany

Ofa Bamberg.
More than 90 years' experience
for your health.

Ofa Bamberg is one of the leading manufacturers of medical supplies in Germany. Since the company was established in 1928 we have been placing extremely high demands on our products: maximum comfort and optimum efficacy. As well as medical supports and orthoses, Ofa Bamberg offers a comprehensive range of medical compression stockings and preventive health and travel stockings.

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Dear user,

We are pleased that you have chosen a product from Ofa Bamberg and would like to thank you for putting your trust in us. Please take a moment to read through this product information carefully to ensure optimum effectiveness of your Dynamics product.

Ofa Bamberg wishes you a speedy recovery!

Fitting instructions

The fitting instructions in pictures can be found on the fold-over page at the beginning of the brochure.

1

To put on the shoe, open all Velcro fasteners and remove the toe cap by undoing the Velcro fasteners at the edge of the shoe. Put your foot in the shoe and close the Velcro fastener over the toes.

2

If required, the toe cap can be fitted to the edge of the shoe again using the Velcro.

3

Close the Velcro tape over the instep in such a way that the tongue of the toecap is beneath it if applicable. Then close the Velcro tape at the ankle by pulling it through the provided eyelet and fastening it. Please ensure that the product is secure but not too tight in order to prevent pinching.

The Dynamics bandage and relief shoes

provide maximum mobility after foot operations, injuries and for other wound processes. Their open design makes them easy to put on, and they can be worn with or without the toe cap. The specially shaped soles of the relief shoes provide targeted pressure reduction in the forefoot or heel area.

Please note

- The initial adjustment and strapping on of the product must be carried out by qualified specialist staff. It might be necessary to make adjustments for uneven leg lengths.
- Do not wear the products directly on injured skin. Daily bandage checking is recommended for patients with polyneuropathy (nerve disease) in the feet.
- Do not drive a car or any other vehicle whilst wearing the shoe.
- If you experience an unpleasant sensation, intense pain or any other problem when wearing the product, then contact your doctor or specialist supplier immediately.
- We use a comprehensive quality assurance system to check our products. However, should there still be reasons for complaint, please contact your specialist supplier.

english

Care instructions

- Wash the product by hand at a maximum temperature of 30° C. Use a mild detergent without softener (e. g. Ofa Clean special detergent) and rinse the product out well. Carefully squeeze the water out without wringing out the product.
- Mould the product into the right shape and air-dry it. Do not dry the product in the dryer, on a radiator or in direct sunlight and do not iron.
- In order to maintain the quality of your product please do not use any chemical cleaning products, bleaches, petrol or fabric softener. These substances can irritate the material of the product.

Indications

Bandage shoe

- after operations and injuries in the forefoot area
- rheumatic feet
- general wounds in the forefoot area, also due to diabetes mellitus

Forefoot relief shoe Varioped

- after operations and injuries in the forefoot area
- general wounds in the forefoot area, also due to diabetes mellitus

Heel relief shoe

- after operations and injuries in the heel area
- general wounds in the heel area, also due to diabetes mellitus
- for relief in the case of fractures in the metatarsal and tarsal area

Side effects

There are currently no known side effects for any part of the body if this product is used correctly. However, if the product is too tight, this can cause local pressure points or constrict blood vessels and nerves.

Material composition

50 % polyester
35 % EVA
10 % polyamide
5 % rubber

Care

-  Hand wash
-  Do not bleach
-  Do not dry clean
-  Do not put into a dryer
-  Do not iron

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Warranty

If used as intended and maintained properly then the product will have a useful lifetime of six months. The product is to be handled carefully in accordance with these instructions for use. Inappropriate use or inappropriate changes to the product can limit the performance, safety and proper functioning of the product and shall lead to an exclusion of claims. The product has been designed for the treatment of one individual patient. It is not designed for usage by a second patient or further patients.