memory®

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Elegant medical compression with skin care

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Sporty and elegant medical compression

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Versatile medical compression with cotton

Medizinische Kompressionsstrümpfe
Medical compression stockings
Bas médicaux de contention
Medische compressiekousen
Calze a compressione graduata per uso medico
Medias de compresión medicinales
Meias clínicas de compressão
Ιατρικές κάλτσες συμπίεσης
Medikal kompresyon çorapları
Zdravotní kompresivní punčochy
Medyczne pończochy uciskowe
Medicīniskās kompresijas zeķes
Meditsiinilised kompressioonsukad
Medicinēs kompresinēs kojīnēs
Ciorapi compresivi medicinali
Medicinske kompresijske nogavice
Gyógyászati kompressziós harisnya
Медицински компресивни чорапи
Medicinske kompressionsstrømper
Medicinska kompressionsstrumpor
Lääkinnälliset kompressiosukat
Medicinske kompresivne čarape
Medicinske kompresivne čarape
Медицинские компрессионные чулки
Медичні компресійні панчохи
الجوارب الطبية الضاغطة
רבי' צחל רפואי
სამედიცინო კომპრესიული წინდები
Բժշկական կոմպրեսիոն գոլպաներ
의료용 압박 스타킹
醫用壓力襪
Эмчилгээний даралтат оймс
Тиббий сиқувчи пайпок

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EN

Elegant medical compression

Memory compression stockings are the perfect combination of modern elegance and a high level of comfort. Delicate shine, subtle transparency and an attractive colour range make this collection a fashionable experience.

Elegant medical compression, including skin care

Memory Aloe Vera combines the modern elegance of Memory compression stockings with natural skin care. Aloe Vera extracts and vitamin E naturally moisturise the skin every time you wear the stockings. Functional yarns and a sophisticated knitting technique ensure a pleasantly soft feel. Memory Aloe Vera: beautiful, healthy and well-groomed legs all day long.

Sporty, elegant and medically effective

Memory Men – the compression stocking for men. Thanks to the high cotton content, the knee high stocking is particularly soft and comfortable to wear. Its subtle look makes it suitable for business. It also goes perfectly with a sports outfit. The colour palette consists of two standard and two mottled shades. Available in standard or made-to-measure.

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EN

The versatile medical compression

The comfortable all-rounder wear in round-knitted: opaque and modern, with an exact fit and soft feel! Lastofa offers reliable support for men and women. In many variations and with thoughtful details. Also suitable for soft connective tissue and advanced venous insufficiency. In compression class 1 and 2 as standard and made-to-measure products.

The versatile medical compression with cotton

The robust and skin-friendly compression stocking with cotton directly on the skin. Especially suitable for patients who suffer from allergies and patients who prefer a robust quality. In compression class 2 and 3 as standard and made-to-measure products.

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Dear User,

We are delighted that you have chosen Ofa compression stockings and thank you for your trust in our products. Please take a moment to read this product information carefully. Getting the best results from your compression therapy!

Intended use

Compression stockings are used to treat phlebo-logical and lymphological disorders of the arms and legs, as well as lipoedema. Different products are used depending on the respective medical condition.

How medical compression stockings work

Products tailored to the respective medical condition and controlled pressure accelerate blood flow, improve microcirculation in the tissue and increase lymph flow.

Important information

Only wear medical compression stockings once you have consulted your doctor. Wearing them every day and attending regular medical check-ups will ensure your treatment is successful! The stockings must be dispensed by medically trained personnel. Your specialist retailer will also advise you on how to put your compression stockings on correctly. If you experience acute leg pain or skin irritation while wearing the stockings, please seek medical advice without delay. Skin reactions in the area of adhesive strips can be prevented by slightly repositioning the adhesive strip several times throughout the day. Please consult a doctor, if wearing the stockings is to be combined with other medical aids. We test our products as part of a comprehensive quality assurance process. Please contact your specialist retailer if you nevertheless have any complaints. Do not repair damage yourself. Return the stocking to the specialist retailer where it was purchased. Fibre knots, which can form due to friction against other items of clothing, do not reduce the quality of the stocking.

Care instructions

The product is suitable for immediate use. We recommend washing it before wearing for the first time. Thereafter, please wash daily by hand or by selecting the delicate cycle of your washing machine. Wash the stockings separately in mild detergent without fabric softener (e.g. Ofa Clean special detergent) at a maximum temperature of 40°C and rinse them thoroughly. The stockings can be spun gently, but please do not wring them out. Air dry your compression stockings or dry them on the gentle cycle of your tumble dryer, but do not place them on the radiator or in direct sunlight. Never iron the stockings. Please do not use chemical cleaning agents, petrol or fabric softener so as to maintain the effectiveness of your stockings. These substances attack the fabric. Compression stockings are largely resistant to fats and oils, but should not come into contact with ointments. Stockings with silicone-coated adhesive bands should generally not come into contact with solvents, ointments, moisturising washing lotions or body creams.

Indications (see also guideline on medical compression therapy AWMF 037/005, as of 12/18)

- Improvement in venous symptoms and quality of life in chronic venous disorders
- Prevention and treatment of venous oedema
- Prevention and treatment of venous skin changes
- Eczema and pigmentation
- Dermatoliposclerosis and atrophie blanche
- Therapy and pain reduction for venous leg ulcers, including recurrence prevention
- Treatment of arterial and venous leg ulcers (see contraindications!)
- Varicose veins
- Initial phase after varicose vein treatment
- Functional venous insufficiency
- Venous malformations
- Venous thrombosis
- Condition after thrombosis
- Post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Lymphoedema
- Oedema during pregnancy
- Post-traumatic/post-operative oedema
- Cyclical idiopathic oedema
- Lipedema from stage II onwards
- Congestion due to immobility (arthrogenic congestion syndrome, paresis and partial paresis of the extremities)
- Work-related oedema (standing or sitting occupations)

- Drug-induced oedema, if no change is possible
- Obesity with functional venous insufficiency
- Inflammatory dermatoses of the legs
- Nausea, dizziness and congestion during pregnancy
- Condition after burns
- Scar treatment

Contraindications

Absolute contraindications:

- Advanced peripheral arterial occlusive disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mm Hg, toe pressure < 30 mm Hg or TcPO₂ < 20 mm Hg arch)
- Decompensated heart failure

- Septic phlebitis
- Phlegmasia coerulea dolens

Risks to be considered

(Therapy decision based on benefits/ risk assessment)

- Weeping dermatoses
- Intolerance to compression stocking materials
- Severe sensory disturbances in the extremities
- Advanced peripheral neuropathy (e.g. diabetes mellitus)
- Chronic polyarthritis

No liability shall be accepted in the event of failure to react appropriately to these contraindications.

Risks and side effects

Medical compression stockings can cause skin necrosis and pressure damage to peripheral nerves. People with sensitive skin may experience itching and skin problems beneath compression stockings. Adequate skin care is, therefore, recommended.

If you experience any of the following symptoms, please remove the stockings without delay and consult your doctor about continuing the therapy. Blue or white discolouration of the toes, abnormal sensations and numbness, increasing pain, shortness of breath and sweating, acute movement restrictions.

Warranty

The shelf life of medical compression stockings is limited to 3 years when properly stored. The wearing time, which is 6 months at most in the case of regular use and proper care, is also to be considered. The products shall be excluded from the warranty once this period has lapsed. As a matter of principle, defects attributable to normal wear-and-tear or improper use are excluded from the warranty.

Improper use also includes subsequent modifications to the product that have not been performed by the manufacturer. Slight colour variations, which may arise due to the individual dyeing process, are not considered to be a defect. Such goods are, therefore, excluded from exchanges and returns. The product is intended for use on a single patient. The manufacturer's product liability shall expire if the product is passed on to other persons for reuse.

Disposal

At the end of its service life, please dispose of the product in accordance with local regulations.

Reporting obligation

Due to legal regulations, patients and users undertake to report without delay to the manufacturer and the competent national authority (in Germany, the BfArM) any serious incidents related to the product that have or could have led to a significant deterioration in health or death.

english

Otherwise, the adhesive properties of the adhesive strips may be permanently damaged (leg hair and dry skin can also impair the function of the adhesive strips). Store the stockings in a dry place, protected from sunlight and heat, preferably in their original packaging.

Wash inside out and do not use fabric softener!

-  Wash on a delicate or gentle cycle
-  Do not bleach
-  Dry gently and carefully
-  Do not iron
-  No chemical dry cleaning

Material composition

Polyamide (Polyamid), elastane (Elastan) and, for some products, cotton (Baumwolle) is used. Please refer to the textile label sewn into your stocking for the precise details.

Dressing tips

Put the stockings on first thing in the morning after getting up, while your legs are not yet swollen. Please do not pull the compression stockings upwards by the top or adhesive edge, but rather slide them up with your hands from the bottom. We recommend wearing household rubber gloves to make putting them on easier and to protect the knitwear. Jewellery and long fingernails can damage the stocking. Please also ensure that you take regular care of your feet and wear suitable footwear. Ask your specialist medical retailer about the Ofa Grip special gloves and the practical Ofa dressing and removal aids for compression stockings with open and closed toe.

How to put them on, closed toe

To put the stockings on, turn it inside out as far as the foot. Slip the foot section over your toes and then over the instep.

Now carefully pull the leg section over your heel. Please avoid pulling up the stockings at the end or grip top.

Pull the rest of the stockings up evenly without overstretching it. Finally, adjust the stockings on the leg until there are no wrinkles.

How to put them on, open toe

Putting on your compression stockings with an open toe is much easier with the supplied Ofa dressing aid (100 % polyester). Slip your foot into the dressing aid so that it reaches just above your heel.

Now pull the stocking up until the heel is in the correct position. This is very easy with the Ofa dressing aid.

Pull back the forefoot section of the stocking to remove the dressing aid.

Then pull the dressing aid forward over the tips of your toes.

The forefoot section of the stocking is then rolled forward again correctly. Then pull the rest of the stocking up evenly without overstretching it. Finally, spread the stocking over your leg until it fits without any creases.

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Ofa Bamberg GmbH